

Deluxe™ Instructions

For PVC Tilt-in-Space Bath Chair

Model Numbers:

BCCD-8500S (up to 100 lb. weight capacity)

BCCD-8600M (up to 130 lb. weight capacity)

BCCD-8700L (up to 180 lb. weight capacity)



These products covered by U.S. Patents:
D606,387; D606,388; D606,389; D592,492; 3,795,218

ATTENTION

Your Deluxe Bath Chair has been built to the highest standards of quality to ensure years of superior service. Please take the time to read the informative instruction guide and pay particular attention to the safety information provided.

Here are some precautions you should take to reduce the risk of injury:

1. Read and understand ALL instructions.
2. Determine, with a physician or therapist, the suitability of this product for the intended user.
3. To reduce the risk of injury or drowning, NEVER leave the user unsupervised or unattended while using this product.
4. Check regularly that all tubing, bolts, buckles, and other parts are tight at ALL times.
5. Check regularly that all PVC tubing and connectors are free from cracks or distortion. At the first sign of cracks or distortion, discontinue use immediately.
6. SAVE THESE INSTRUCTIONS.

If you require additional assistance, please contact Inspired by Drive Customer Service at **(800) 454-6612**, or email at **info@inspiredbydrive.com**.

Thank you for choosing Inspired by Drive.

Before You Begin

The following are assembly instructions for each of the different sizes of Inspired by Drive Deluxe™ PVC bath chairs. Follow ONLY the instructions that are specific to your particular model.

Models BCCD-8500S Deluxe Bath Chairs (Small)

The small Deluxe bath chairs are shipped with the frame and seat unattached.

1. Reach inside the seat cover by opening the top flap of the mesh cover.
2. From the inside of the seat, insert the bolts through the pre-drilled holes in the plastic tubing, the through the holes in the plastic frame. The cap nuts should then be screwed to the bolt ends protruding on the outside of the frame.
3. Insert the tube at the rear of the seat back into the frame's larger tube by pressing the metal release button.

Models BCCD-8600M Deluxe Bath Chairs (Medium)

The medium Deluxe bath chairs are shipped with the seat already attached to the frame.

1. Insert the tube at the rear of the seat into the frame's larger tube by pressing the metal release button.
2. Additionally, insert the tube under the seat front into the frame's larger tube by pressing the metal release button.

Models BCCD-8700L Deluxe Bath Chairs (Large)

The large Deluxe bath chair is shipped with the seat and frame unattached.

1. Bolt the seat section to the frame using one bolt on each side.
2. Insert the tube under the seat front into the frame's larger tube by pressing on the metal release button.

Setting the Angle of Recline

1. Push the release button on the upright post. Slide the inner post up or down, until the bottom locks into the desired position.

CE

BS Partnership
41 Dan-y-Coed Road
Cardiff
CF23 6N
UK


Inspired
by *drive*

11724 Willake Street
Santa Fe Springs, CA 90670
Toll Free: 800-454-6612
Direct: 562-282-0244
Fax: 310-305-1718

Website: www.inspiredbydrive.com

Email: info@inspiredbydrive.com

© 2017 Columbia Medical, LLC dba Inspired by Drive

Every Inspired by Drive product comes with our one year manufacturer's warranty. This warranty ensures that when you receive our products, they will arrive free of defects in material and workmanship. We will repair or replace any part or entire product at our discretion for a period of up to one year from the date of delivery to the original purchaser. This warranty does not include cost of inconvenience, shipping, property damage, misuse, abuse, accident, or similar incidents. The warranty will not apply if the product has been modified, tampered with, repaired by unauthorized individuals or assembled incorrectly. Custom orders are non-refundable and non-returnable.