

# Aquanaut™




*"... the new wave  
in toileting"*

*Designed specifically  
for children who  
need help maintaining  
a sitting position.*

*Contact your dealer  
or therapist with any  
questions about the  
suitability of Aquanaut  
for your child.*

**VASI** Variety Ability  
Systems Inc.

2 Kelvin Avenue, Unit 3  
Toronto, Ontario  
Canada M4C 5C8  
Tel: (416) 698-1415  
Tel: 1-800-891-4514  
Fax: (416) 698-5860  
email:  
mmifsud@vasi.on.ca

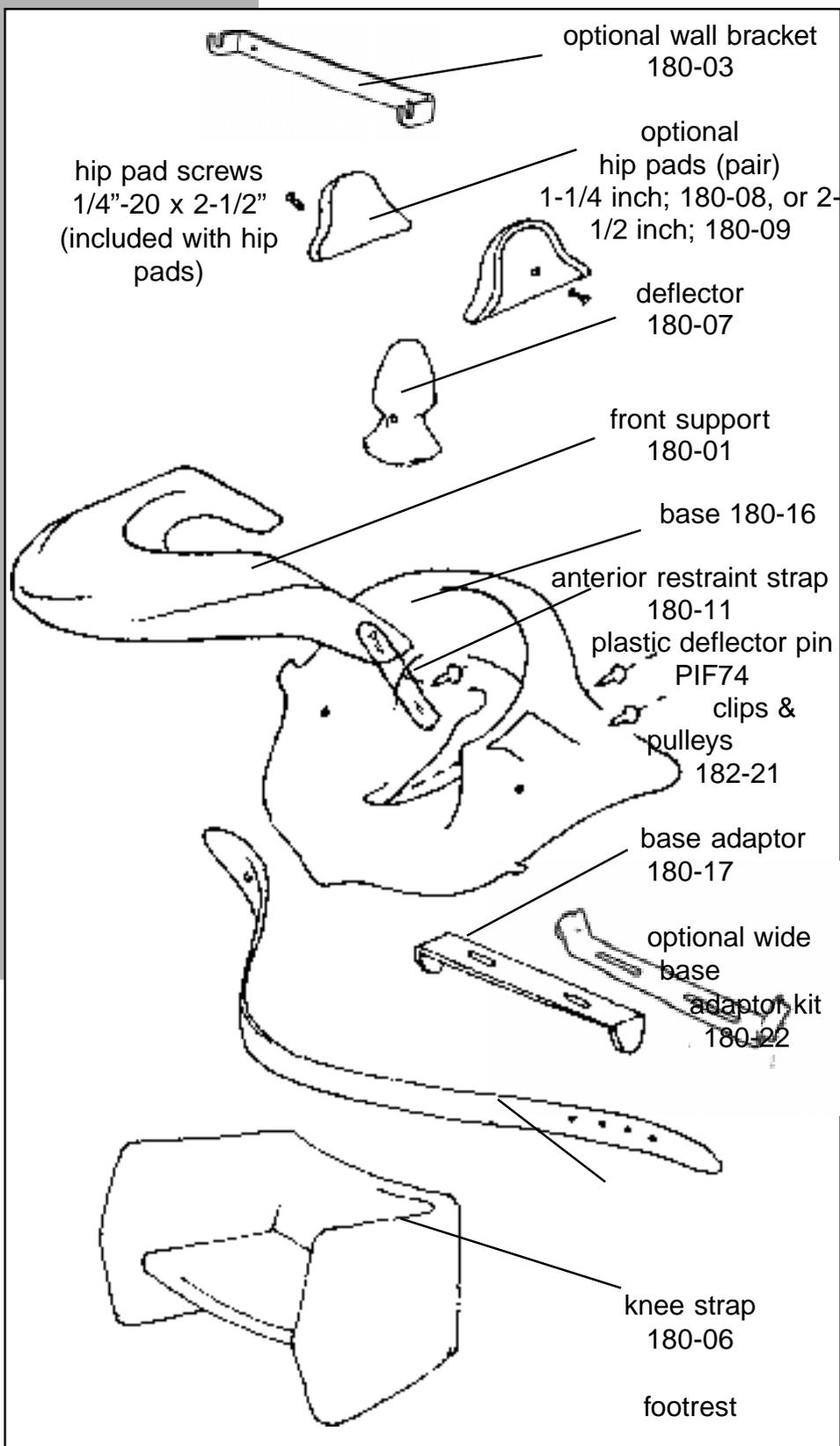

# Aquanaut

## Set Up

The Aquanaut toilet seat is not suitable for all children. Consult your dealer with any questions about this product's suitability for your child.

1.



Insert knee strap into groove in underside of base.

2.


If required, insert deflector into aperture in base, after removing deflector pin.



Re-insert pin through holes.


3.

Standard adaptor  
Note orientation of tabs face downwards

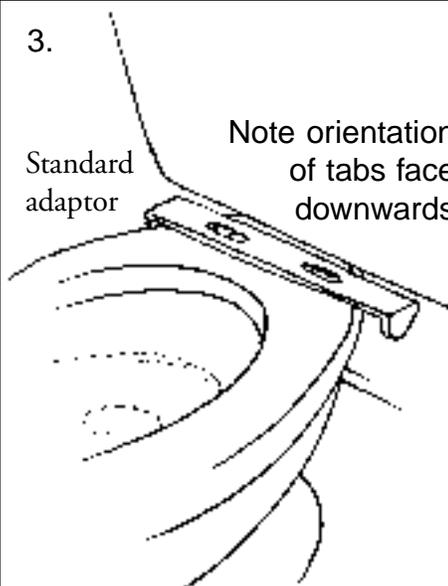

Remove toilet seat and centre adaptor over holes. Install either the standard adaptor with tabs facing down or the wide base adaptor with tabs facing

4.

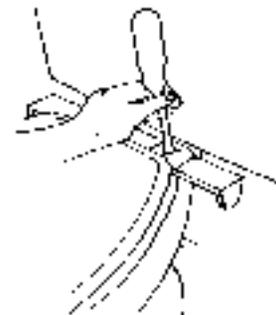


Replace toilet seat and tighten screws so adaptor is snug. Install bumpers under the base of Aquanaut if using the wide base adaptor. Screw the two bumpers into plastic base positioning bumpers beside and towards the front of the raised plastic areas as shown.



Do not over-tighten screws

Note orientation of tabs face upwards

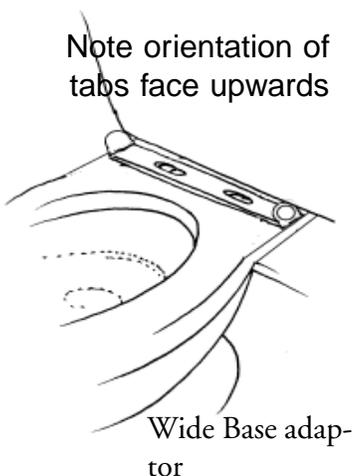

Wide Base adaptor

# Aquanaut


## Fitting

1.  
Raise toilet seat.  
Lower base down  
onto toilet bowl.



Tilt base backwards to  
touch toilet seat.

2.  
Slide base backwards to  
catch adaptor tabs.

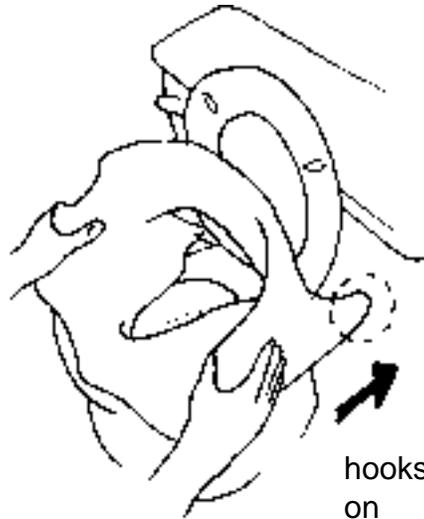


Tilt forward to lock.

*Properly positioned, the seat base of Aquanaut will not move more than 1/2 inch (1.2 cm) from side to side, and 1 inch (2.5 cm) from front to back.*

*Consult your dealer if you are not sure about properly installing the seat base.*

*For smaller children, two sizes of hip pad are available, to fit hip widths of about 9 inches (23 cm) to 12 inches (30 cm). Without hip pads, the seat fits children with hip widths up to 14 inches (36 cm).*

*Consult your dealer or therapist for help in fitting your child properly.*

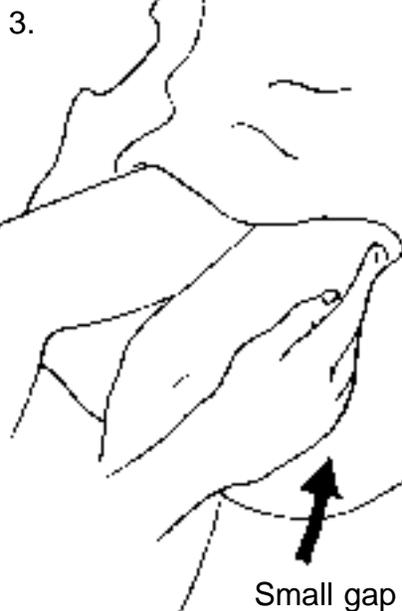

Verify comfortable fit.



If provided, install hip pads  
for smaller children.  
Do not over-tighten screws.

# Aquanaut

## Positioning your child

*If you have any questions about handling your child properly while using Aquanaut, speak to your dealer and your child's therapist.*

*Take care to avoid back problems when lifting. Bend your knees as you lift or lower your child.*

*Never allow any child to use Aquanaut without the front support in position.*




Hold your child in a sitting position, hips and knees flexed.



Keeping this sitting position, lower your child into the seat.



Support your child with one hand.

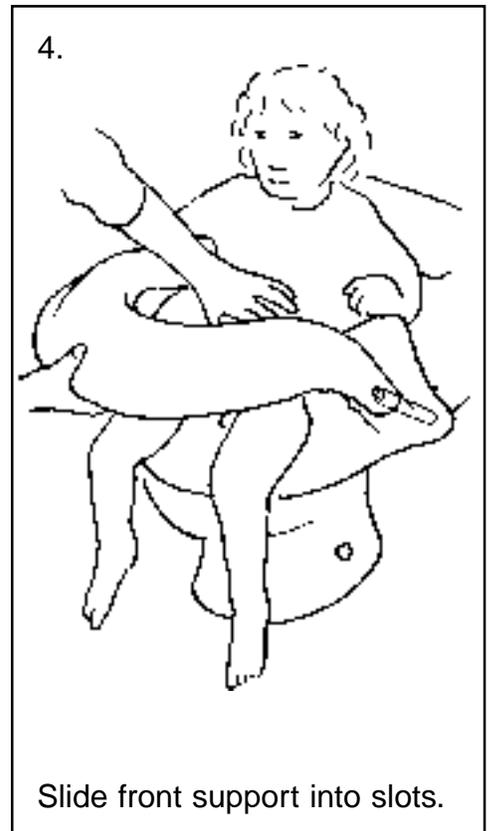




Slide front support into slots.

## Adjusting Seat

*Never leave any child unsupervised on Aquanaut. Stay with your child at all times.*

*Unsupervised children may slip downward, fall sideways, or go into backwards extension, striking their heads on hard surfaces.*

*Some children may prefer not to use the footrest. Consult your therapist for advice.*

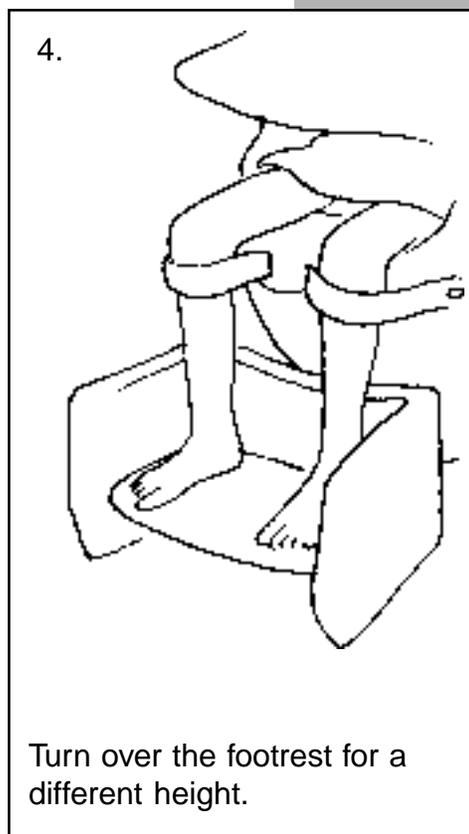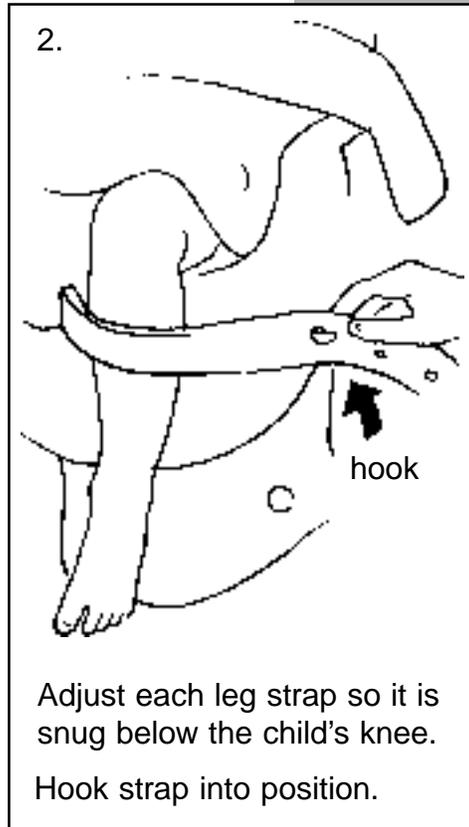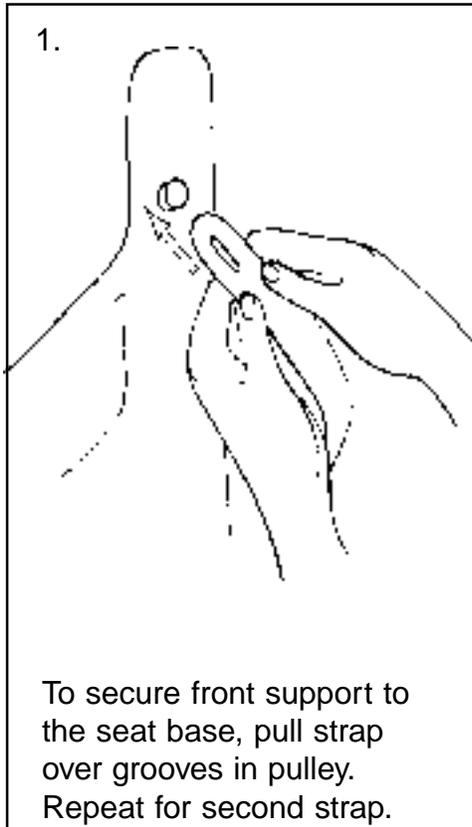




# Aquanaut

## Storing and cleaning





Use a mild, non-abrasive antibacterial liquid soap.

Do not use abrasive cleaners. Abrasives damage plastic surfaces.

Harsh soap residues may contribute to skin rashes in some children.

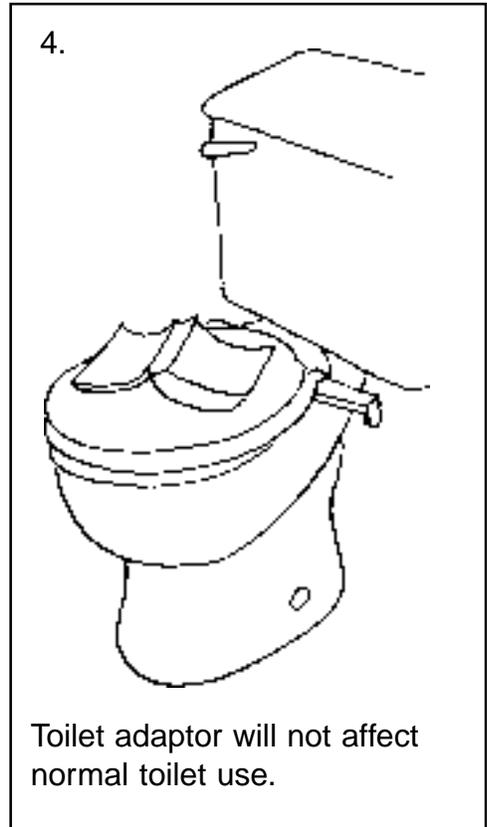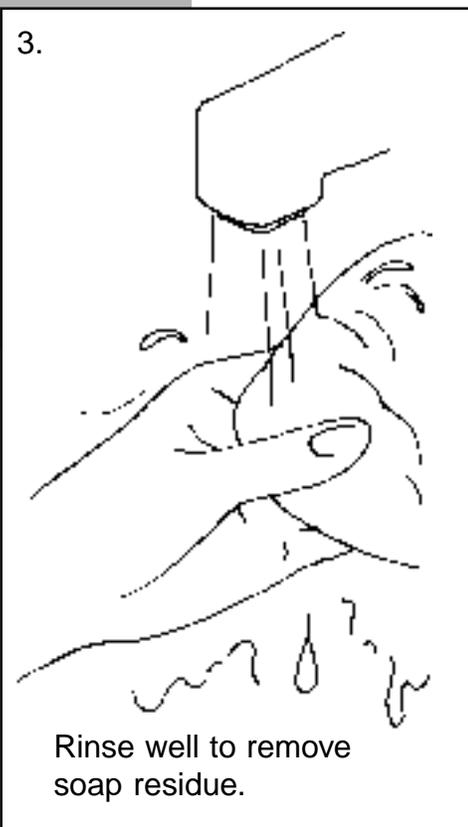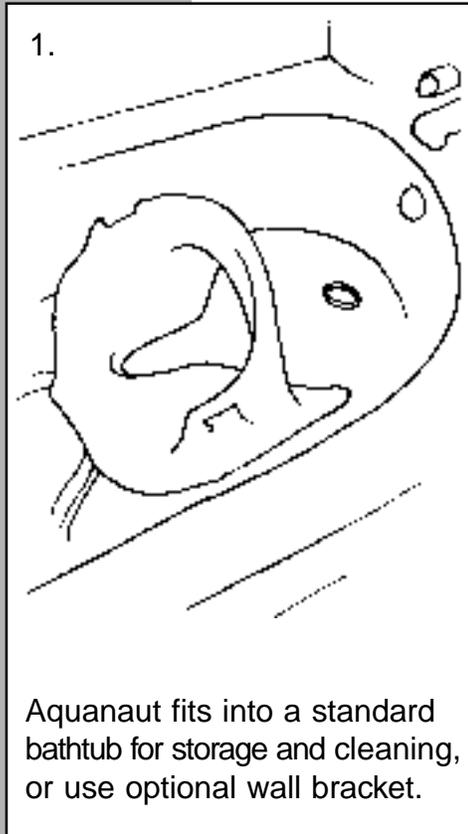

# Aquanaut

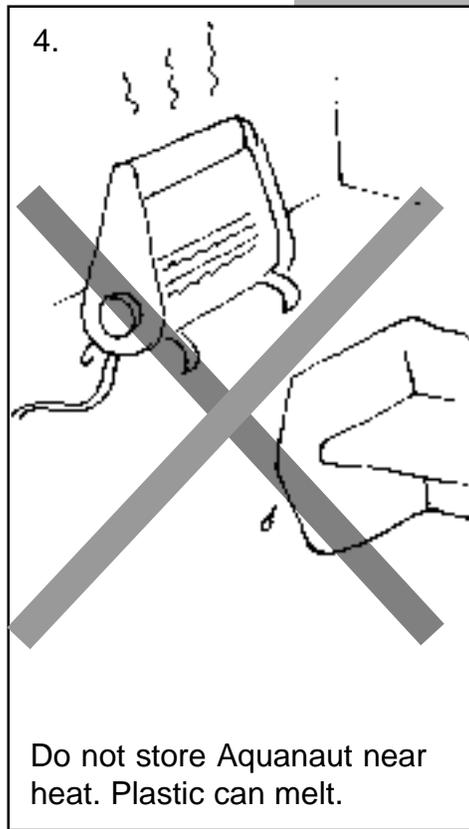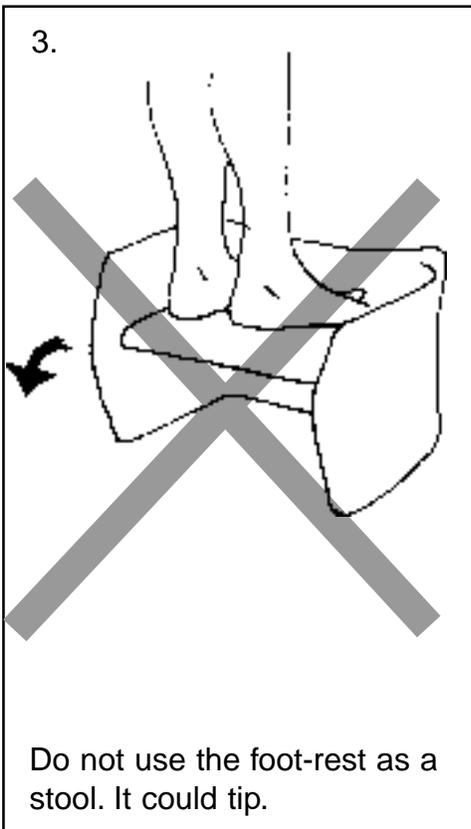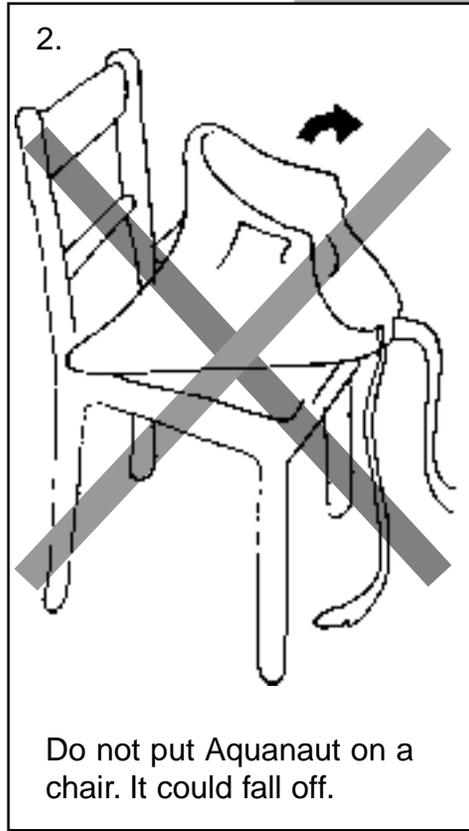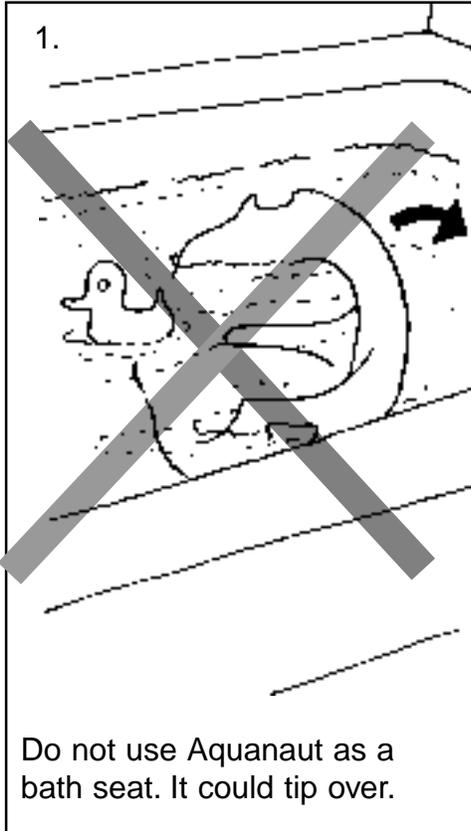

## DO NOT...

*Improper use of Aquanaut could result in injuries to children.*

*Do not use Aquanaut for anything other than its intended use.*

*Do not use or store Aquanaut near heat, sparks, or flames.*

*Do not leave your child in Aquanaut unattended.*

*Aquanaut is not recommended if your child has any or all of the following:*

- *uncontrolled movements*
- *high extensor tone*
- *postural insecurity.*

*Check with your child's therapist if you are unsure.*


# Aquanaut

## Options and Spare Parts


A wall bracket provides convenient storage for Aquanaut off the floor.

AQUANAUT and VASI are trademarks of Variety Ability Systems, Inc., Toronto, Canada.


US Patent 5778460.  
Foreign design patents pending.

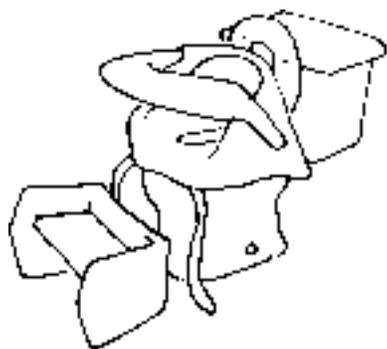

1. Choose a location which allows sufficient space for Aquanaut storage, and does not interfere with other bathroom activities.

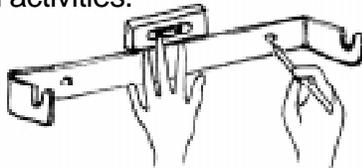


Make sure the bracket is the right way up, level it, and mark the centres of the holes. Drill two 1/2" holes in dry wall.

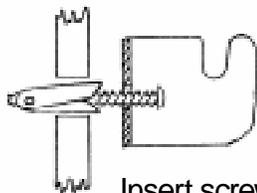


Insert screws through the holes in the bracket, and hand-tighten spring toggle clamps on the ends. One at a time, push the assembled spring clamps through the holes in the dry wall.

2 Tighten screws till spring clamps are pulled tight.

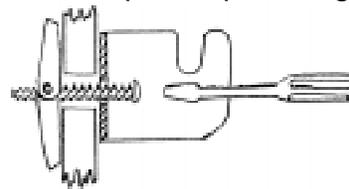

Leave the front support in place, and hang Aquanaut from the bracket by the side tabs.

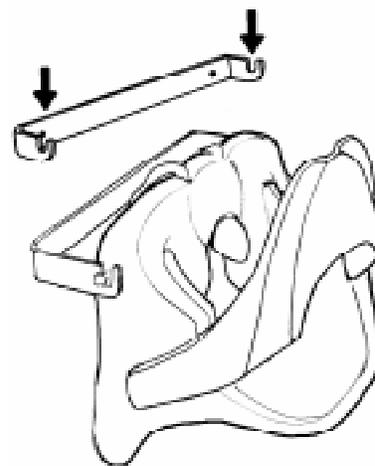

### OPTIONS AND SPARE PARTS

Product Number	Product Description
180-01	front support
180-02	footrest
180-03(optional)	wall bracket
180-17	base adaptor
180-06	knee strap
180-07	deflector
180-08(optional)	hip pad 1-1/4" with mounting screws (pair)
180-09(optional)	hip pad 2-1/2" with mounting screws (pair)
180-11	anterior restraint strap (2 required)
180-16	base
PIF74-10	deflector pin (pkg. of 10)
180-20	machine screws 1/4" -20 x 2-1/2" (pkg. of 10)
180-22(optional)	wide base adaptor kit with adaptor & bumpers
182-21	clips and pulleys for anterior support restraint

Copyright © 2002

**VASI** Variety Ability Systems Inc.

2 Kelvin Avenue, Unit 3  
Toronto, Ontario Canada M4C 5C8  
Tel: (416) 698-1415 Fax: (416) 698-5860