



USER GUIDE

BUGGY CORZO Xcountry



Dear customers,

BUGGY CORZO and BUGGY CORZO Xcountry are designed for short stay of an occupant. With the design and especially unique construction are ranked among the best from their category. Prior the first usage of the buggy, please, read carefully this user guide. In case you are not fully sure about the proper usage of the product after reading of the user guide feel free to contact your local dealer. Retain the user guide for future reference.

SAFETY NOTES

- please read this instruction booklet fully prior to use
- for safety reason it is necessary to check prior using of buggy, that rear brace (seat brace - size 38), safety levers are properly tightened and the spring button of the seat and handles quick releases are properly positioned
- expert medical advice regarding the suitability of this product must be sought before use
- the attendant should be familiar with the operation of the buggy - if in doubt, stop and read this instruction booklet
- before setting off, ensure the child is comfortable and that clothes and rugs etc. are tucked in, so they do not catch in the wheels
- always apply the parking brakes when the buggy is stationary and when getting in and out the buggy
- never tip the buggy forward at a kerb
- the buggy **MUST NOT** be used as a seat for a child while traveling in motor vehicle or bus
- the attendant must take precautions against fire risk. In buildings, check the location of fire exits and ensure procedures are understood.
- do not let children play with or hang on the buggy
- do not use your buggy in a manner for which it is not designed
- PATRON Bohemia s. r. o. cannot accept responsibility for the use of accessories other than those supplied by this firm
- during the drive keep full attention, to avoid any dangerous situation or collision
- do not allow the child to climb unassisted into and out the buggy
- always apply the parking brakes when getting in and out the buggy
- do not overload this buggy with additional children, goods or accessories other than those supplied by PATRON Bohemia s. r. o.
- ensure the child always wears a correctly fitted and adjusted safety belt. The attendant must know how to fit, adjust and release accessories
- do not stand on the footrest
- never leave child unattended in the buggy, even for the shortest time
- by the using and manipulation with buggy act accordingly not to cause any injuries to the occupant, yourself or any other person
- PATRON Bohemia s. r. o. is not responsible for any loses and injuries caused by incorrect manipulation and using of the product.



WARNING NOTES

- ① BAGS ATTACHED TO THE HANDLE AFFECTS THE STABILITY OF THE BUGGY!
- ② NEVER LEAVE YOUR CHILD UNATTENDED!
- ③ RECOMMENDED AIR PRESSURE: front/rear - 230/230 kPa



MAXIMUM WEIGHT OF OCCUPANT



MAXIMUM LOAD OF THE BUGGY

OPERATING INSTRUCTION

Opening the buggy:

stand buggy on wheels, unlock storage clip, grasp one handle and push down on the opposite seat tube. Press down firmly on seat tubes to ensure the buggy is completely opened. Use foot to lock the rear brace into a straight position. Wrap the armrest padding around the armrests. Mount the footrest on the buggy. (Lock the seat brace - size - 38)

Footrest:

install - put on the floor almost decomposed braked stroller and release both safety levers. Take the footrest, press spring button placed at the rear top of the tube and pull both tubes into the both footrest attachments until they are locked in the lowest position by the rear spring button.

height adjustment - release both safety levers, pull both spring buttons of the footrest and adjust it into desired position. Tighten both safety levers to secure the footrest in position.

uninstall - put on the floor almost decomposed braked stroller. Place the footrest into its lowest position, press the rear spring buttons and pull out the footrests both side tubes at the same time.

Seat depth adjustment:

the seat should support the tights on their whole length. To change the seat depth undo the velcro panel connecting the seat and backrest and partially fold the buggy to relieve fabric tension. Grasp end of seat tube, press seat spring button and move seat tube until spring button relocates into the desired hole. Repeat for other side of seat. Fully open the buggy and fasten the velcro panel connecting the seat and backrest.

Backrest and handles height adjustment:

undo the velcro straps of the backrest extension and partially fold the buggy to relieve fabric tension. Grasp the handle, release handle safety lever, pull out the handle quick release, move the handle until it relocates into desired position. Release the handle quick release by moving the handle. Tighten the handle safety lever. Repeat for other handle and fully open the buggy. Fasten all velcro straps of the backrest extension.

Brakes:

lock - pull the brake lever down to lock the wheel

unlock - pull the brake lever up to unlock the wheel

Five-point belt:

release all plastic quick fasteners (fork type), place the child into the buggy and adjust of desired lengths the belt. Choose the hole in backrest to adjust the proper position of the shoulders straps. The hole should be level with or higher than the top of the shoulders. The shoulders straps must be mounted on backrest and backrest extension at once when possible.

Getting into the Buggy:

ensure the parking brakes are on, to make access easier, the footrest can be taken off; if possible, place the back of the buggy against a wall or immovable object (alternatively, another helper may hold the buggy steady); place child carefully into the buggy, fasten safety belt and fit the footrest. The child must always feel comfortable.

Getting out the Buggy:

ensure the parking brakes are on; if possible, place the back of the buggy against a wall or immovable object (alternatively, another helper may hold the pushchair steady); unfasten safety belt and lift the child carefully.

Negotiating the Kerb or Single Step:

down - take the rear wheels to the edge of the kerb, hold the handles firmly and tip the buggy slightly backwards. Balance on the rear wheels, lower the buggy down the kerb on its rear wheels. Never tip the buggy forward at a kerb.

up - take the rear wheels to the kerb. With a firm grip on the handles, tip the buggy backwards and place the rear wheels on top of the kerb. Position the front wheels at the kerb bottom and lift to the upper level.

Negotiating Steps:

- make use of ramps and lifts when they are available
- where possible steps should be avoided, if there is no alternative to negotiating steps then ensure at least one helper is available to give assistance
- do not attempt to take the buggy up or down a flight of steps unless you and your helper are sure you have control of the buggy and can take the weight of its occupant
- during this operation, the attendant and helper must be well balanced and have secure grip on the buggy

- the attendants hands must be inside the handle straps at all time
- take rests in regular intervals
- do not lift the buggy with footrest

going down steps - position the buggy squarely with the front wheels on the top step, tilt the handle backward and push the buggy forward until the rear wheels are at the top step; the helper stands in front, steadies and lowers the buggy by gripping the front frame; take the weight and lower the rear wheels down the step, continue step by step to the bottom. (picture no. 1)

going up steps - position the buggy squarely with the rear wheels against the bottom step, tilt the buggy backward and raise to the next step, the helper stands in front, steadies and lifts the buggy by gripping the front frame, continue step by step to the top (picture no. 2)

Folding the buggy:

unfasten the dry zipper of armrest upholstery, unlock rear brace by pushing up on centre of brace. (Unlock the seat brace - size 38) Grasp one handle and pull up the seat tube with other hand. Lock the storage clip. Is not necessary to uninstall the footrest for folding the buggy. (Picture no.3)

Upholstery:

the seat, seat reinforcement, backrest and backrest extension are not detachable. You could clean them by warm water and mild detergent.


Armrests:

undo the velcro and button to uninstall the armrests upholsteries.


Wash by hands on max. 40°C with mild detergent.

Seat reinforcement:


you could increase or decrease the seat rigidity by adjustment of the seat reinforcement. Lengthen the belts of seat reinforcement to decrease the rigidity. Shorten the belts of seat reinforcement to increase the rigidity. Never change the seat width by adjusting of the seat rigidity.

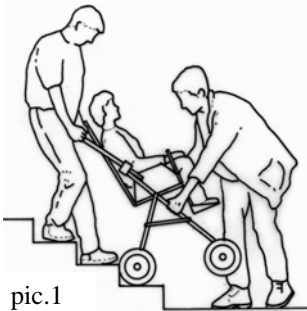

pic.1

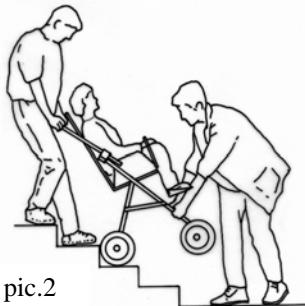

pic.2

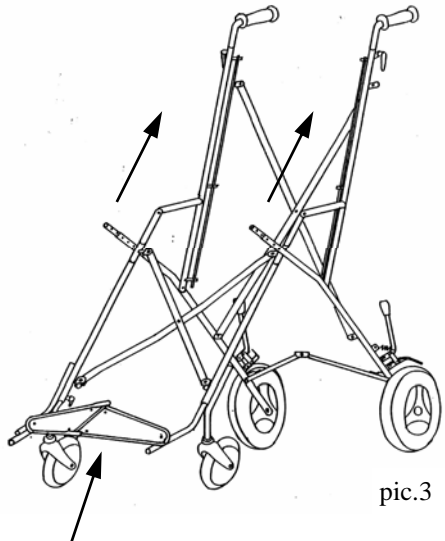


pic.3

MAINTENANCE INSTRUCTION

- your buggy has been designed to a high standard and with correct use and maintenance will give many years of trouble free service
- regularly inspect locking devices, catches, footrest, joints fixings to ensure they are secure and in full working order
- brakes, wheels, tires are subjects to wear and should be replaced if necessary by a repairer approved by the buggy supplier
- we recommend that the buggy is checked by an approved repairer every 12 month.
- if the buggy is damaged or in need of repair, the approved repairer will be able to advise on what is required.
- alternatively, PATRON Bohemia s. r. o. Customer department is available for product information.
- only PATRON Bohemia s. r. o. replacement parts should be used on this buggy.
- should parts of your buggy become stiff or difficult to operate (or not later than every 6 month), ensure that dirt is not the cause, apply a light application of a spray lubricant e. g. WD 40 (do not use oil or grease). Ensure that the lubricant penetrates between the components but do not over spray as this may encourage the accumulation of dirt. Wipe off any excess spray. Regular lubrication of moving parts will extend the life of your buggy and take opening and folding easier.
- make sure that you are familiar with the operation of the buggy. It should always open and fold easily. If it does not, do not force the mechanism - stop and read the instructions.
- wheels, plastic and metal parts may be sponged clean with warm water and mild detergent.
- the seat fabric can be washed by hands (max. 30°C).
- never clean with abrasive, ammonia based, bleach based or spirit type cleaners.
- after exposure to damp conditions, dry off with a soft cloth and leave fully open in warm environment.
- store your buggy in a dry, safe place.

PATRON Bohemia s. r. o. reserves the right to make design changes to any of their products without prior notice as a part of their continuous development program.

SCHEMA

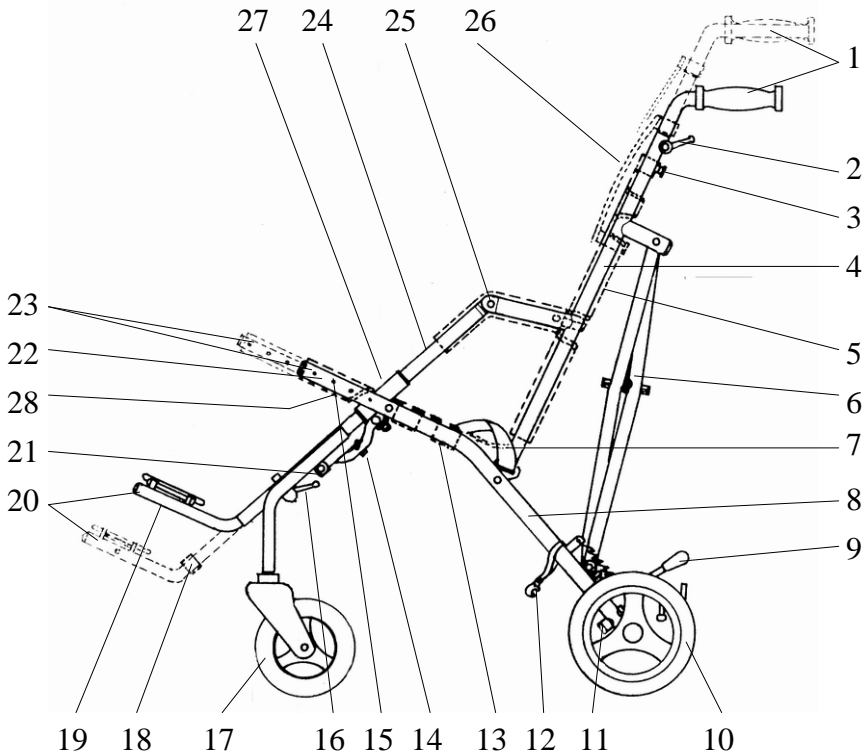

DESCRIPTION

- | | |
|-------------------------------|------------------------------|
| 1 handle | 16 footrest adjusting lever |
| 2 handle adjusting lever | 17 front wheel |
| 3 handle adjusting safety pin | 18 calf rest |
| 4 Backrest frame | 19 footrest safety |
| 5 Backrest | 20 footrest |
| 6 cross-frame | 21 arrestment device spring |
| 7 safety belt | 22 seat |
| 8 seat frame | 23 extension seat adjustment |
| 9 parking brake | 24 side frame |
| 10 rear wheel | 25 side frame cover |
| 11 back safety crossbar | 26 backrest height extension |
| 12 transport safety-catch | 27 frame |
| 13 seat reinforcement | 28 seat clip reinforcement |
| 14 seat crossbar | |
| 15 seat safety pin | |

BASIC TECHNICAL DATA

Basic technical data:	CRX30	CRX34	CRX38	CRX42
Backrest height [mm]	490-560	560-660	680-780	680-780
Seat depth [mm]	260-340	300-380	350-400	350-400
Seat width [mm]	300	340	380	420
Seat to footrest [mm]	220-370	280-450	290-470	330-500
Backrest angle	110°	109°, 112°	109°, 112°	106°, 110°
Seat angle	20°	22°	22°	20°
Total length [mm]	1130	1140	1170	1220
Total width [mm]	560	630	670	720
Total height /with handle [mm]	1020-1120	980-1070	1170-1260	1210-1320
Brake	(foot oper.)	(foot oper.)	(foot oper.)	(foot oper.)
Weight kg	11,5	12,0	13,5	14,0
Maximum load kg	45,0	55,0	75,0	75,0
Folded stroller length [mm]	1200	1200	1200	1250
Folded stroller width [mm]	350	350	350	350
Folded stroller height [mm]	360	360	360	340

WARRANTY SHEET

Final inspection:

Name:

Signature:

Identification label:

**The product is under warranty for period of
24 months after date of purchase.
The warranty covers faulty material and
processing defects.**

Warranty conditions:

1. The product can be used entirely for the purpose it is designed and the instructions in the user guide must be followed.
2. The product has to be properly used, cleaned and stored in accordance to the user guide instructions.
3. The warranty does not cover: failures caused by overloading of the products, forced damage, unauthorized changes made on the device and/or its accessories and natural wear and tear of parts, especially upholstery dirt and damage, wear of plastic parts, brakes, tyre profile and tyre puncture.
4. The warranty claim has to be applied by the seller or authorised service agent of the device.
5. The properly filled warranty sheet and purchase confirmation must be declared by the warranty application.
6. The claim procedure can be started when the device is delivered complete and in origin packing, carefully packed and secured against damage. The owner is committed to deliver the device cleaned, washed and hygienic unobjectionable according general hygienic requirements. The basic maintenance and washing cannot be expected or even requested in the claim.
7. The warranty period does not stand for usage period, that means for the period when by correct use and maintenance the device in accordance to its features, purpose and construction could be used.
8. Neither producer nor its agent will be liable for any losses and injuries caused by manipulation and usage of the product in contradiction of the "User Guide" instructions.
9. The warranty is terminated whenever the following conditions arise: the user fails to follow the instruction mentioned in this user guide and manufacturer's storage and maintenance suggested procedures, or whenever the product is serviced by any other party except by the authorised service centres, or when the identification label is not stucked on the stroller and also in the warranty sheet.

Purchase date (day, month, year):

Name (type), phone number, stamp, signature of seller:

____.____.200__