

# first class chair support kit accessory

item# FC 8025 & FC 8025L  
(for use with item# FC 2000 & FC 4000)

includes trunk harness,  
abductor, & lateral supports



EU Authorized Representative  
Drive International, LLC  
Globe Lane - Stockport  
DUKINFIELD SK16 4RE, GB

*Wenzel* **Yte** *Re/hab*

drive  
[WWW.DRIVEMEDICAL.COM](http://WWW.DRIVEMEDICAL.COM)

# support kit accessory

## trunk harness

1. Attach shoulder straps to Velcro strips on back of chair (figure 1). Adjust to desired height.
2. Open harness belt and loop around the back of the chair. Adjust and close to desired width (figure 1).
3. Upon initial use, adjust belt clip (A) to appropriate size (figure 2).
4. Use zipper to secure user and provide anterior chest support.

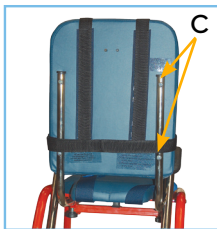


figure 1



figure 2

## abductor

1. Insert abductor into square receptacle under seat (figure 3).
2. Adjust abductor to desired depth and tighten knob (B) to secure.

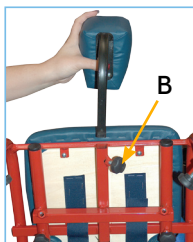


figure 3

# support kit accessory

## lateral support

1. Remove screws (C) (figure 1) on back support.
2. Mount lateral support bar. Ensure that the black washer is in between the back support and the lateral support bar (figure 4).
3. Loosen height/width adjust knob (D) (figure 5) and insert lateral support.
4. Adjust height by raising or lowering lateral support bracket (E) to desired height and tightening knob to secure in place.
5. To adjust width, loosen knob (D), push/pull lateral pad to desired width and tighten knob to secure in place.



figure 4

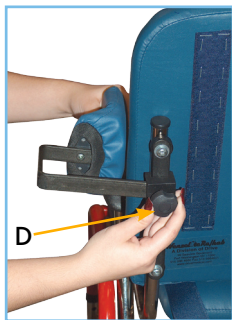


figure 5

**drive**  
WWW.DRIVEMEDICAL.COM

Wenzel*ite*Re/hab

# two year limited warranty

Your Drive brand product is warranted to be free of defects in materials and workmanship for two years from the date of the original consumer purchase.

This device was built to exacting standards and carefully inspected prior to shipment. This Two Year Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of dependable service.

In the event of a defect covered by this warranty, we will, at our option, repair or replace the device.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Drive device or this warranty, please contact an authorized Drive dealer.



*Wenzel*iteRe/hab

© 2006 Medical Depot, Inc. All rights reserved.

Drive is a trademark of Medical Depot, Inc.

Port Washington N.Y. 11050 USA Made in China