



Instructions for using Flip2Sit™ 2-in-1 activity seat

Please read all warnings and instructions below.
Save instructions for future use.



WARNINGS

- **To avoid falls, always secure the product to a chair using the safety belts provided.**
- **Always provide a table or desk for your child when using the product on a chair.**
- **Do not use product on a stool, folding chair, unstable chair or on top of a table.**
- **Do not use as a bath seat or a car seat.**
- **Do not attempt to remove screws from the safety belts as this may damage the product.**
- **Never leave your child without adult supervision.**
- **Do not lift your child in the product.**

IMPORTANT: Please inspect this product for damage before each use. DO NOT USE IF ANY PARTS ARE BROKEN OR MISSING. Contact a VASI customer service representative for replacement parts. (See page 4 for contact information.) Never replace with parts from other sources.

To switch between the two seats

- Place the unoccupied product on the floor or chair.
- Hold the top of the back with one hand and hold the seat with the other.
- Pull up on the back so it disconnects from the seat (Figure 1).
- Flip the seat over with the other hand.
- Replace the back. Push downward until it is firmly attached to the seat.



Fig. 1



Fig. 2

To tighten belts

- Push the end of the belt through the buckle so it forms a loop.
- Pull the free end of the belt away from the buckle (Figure 2).
- Pull the belt webbing through the plastic fastener to hold it in place.
- Repeat steps to tighten further.

To loosen the belt

- Push the end of the belt through the buckle so it forms a loop as above.
- Pull the long end of the belt away from the buckle.
- Pull the belt webbing through the plastic fastener to hold it in place.
- Repeat steps to loosen the belt further.



Fig. 3



Fig. 4

To attach to a chair

- Place the seat upright on the chair (Figure 3).
- Release the buckles on the back and seat.
- Position the seat in the preferred location on the chair.
- Wrap the back safety belt around the back of the chair. Connect the buckle (Figure 4).
- Pull the loose end of the belt until it is snug.



Fig. 5

- Wrap the seat safety belt around the bottom of the chair (Figure 5).
- Connect the buckle and be sure you hear a 'click.'
- Pull the loose end of the belt to hold the seat snugly on the chair.
- Check that the safety belts are oriented as shown in Figure 6.
- Check that the seat does not slide on the chair by pulling up and forward on the seat.
- Follow the procedures below for securing your child in the seat.



Fig. 6



Fig. 7

To use as a floor sitter

- Remove the back and flip the seat over so the contoured side of the seat is facing downward.
- Unbuckle the safety belts. Rotate the safety belts so they are pointing toward the back of the seat.
- Connect the buckle and tighten the belts (Figure 7).
- Pull the back safety belt so it is snug against the back.
- Follow the procedures below for securing your child in the seat.

To secure your child

- Make sure the seat is securely attached to a chair or sitting flat on the floor away from obstructions.
- Release thigh strap buckles and move them apart.
- Transfer your child into the seat. Help your child sit up straight before tightening the thigh straps.
- Connect a buckle so the thigh strap wraps around your child's leg on one side. Repeat with the other strap on the other side (Figure 8).
- Pull on the loose end to tighten the strap.

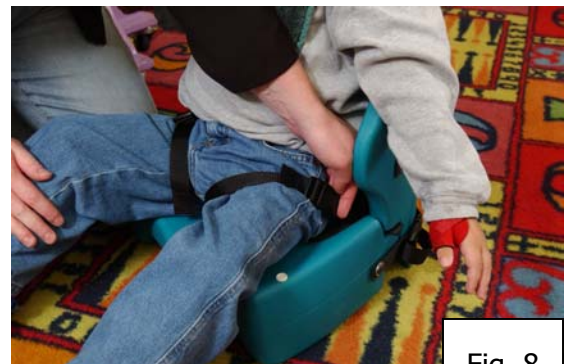


Fig. 8

To clean

- Wipe surfaces with damp cloth using mild soap. Rinse thoroughly with clean water.
- Seat pad and thigh straps can be removed for cleaning.
- Do not remove safety belts from product.
- Do not immerse in water.
- Do not place in dishwasher or washing machine.



Item	Part No.	Quantity	Item	Part No.	Quantity
1 – seat	220-08	1	6 – seat pads	220-03	2
2 – back	220-07	1	7 – belt washers	W-180	8
3 – back safety belt	220-04	1	8 – belt screws	395-046	4
4 – seat safety belt	220-05	1	9 – plugs	245-028	2
5 – thigh straps	220-06	4	10 – anti-slip pads	315-171	8

If you are not completely satisfied with this product,
please contact our customer service representative at:

Variety Ability Systems Incorporated
2 Kelvin Ave, Unit 3, Toronto, Ontario, CANADA, M4C 5C8
Phone: 416-698-1415 FAX (416) 698-5860
e-mail: mmifsud@vasi.on.ca

www.flip2sit.com

© Variety Ability Systems Incorporated 2003 R2