

**TherAdapt<sup>®</sup> Products Inc.**  
(800) 261-4919 phone (866) 892-2478 fax

**Tray Easel**

**www.theradapt.com**

**Please read all of the following information before using this product.**

*TherAdapt<sup>®</sup> Products Inc. has designed and manufactured this product with the well being of the user and caregiver(s) in mind. It is our endeavor to provide products which are attractive, comfortable, useful, highly adjustable, long lasting and safe. As when using any product, precautions must be taken.*



**WARNING:** This product must only be used under the recommendation of a physician and / or licensed therapist and with adult supervision. Never leave user unattended. It is your responsibility to properly use and maintain your Tray Easel. Failure to follow USE and CARE instructions or ASSEMBLE the easel properly as outlined may result in bodily injury.

THE MANUFACTURER IS NOT RESPONSIBLE FOR INJURIES RESULTING FROM ANY CUSTOMIZATION OR MODIFICATION OF THE EQUIPMENT THAT WAS NOT COMPLETED BY THE MANUFACTURER.

**USE:** The Tray Easel is designed for a single purpose: as a freestanding easel. It is to be used on the floor only. No other use is permitted.

**CARE:** Take care to inspect the ongoing condition of all parts of your Tray Easel. Verify that all knobs, bolts, and washers are properly positioned and secure. Contact the manufacturer for replacement or repair of necessary items. All components may be cleaned with mild soap and water, or a gentle sanitizing spray. Do not use abrasive cleaners. Wipe wood components dry after cleaning. Excess amounts of liquid on the wood may damage the finish. Always store indoors.

**WARRANTY:** The manufacturer provides a full one (1) year warranty, beginning the date shipped by the manufacturer. The warranty provides full replacement of all parts against breakage when used in compliance with the manufacturer's USE and CARE instructions. Use or modification beyond the normal capacity or range of adaptability designed into the products will void the warranty. The foregoing is TherAdapt's sole liability for consequential damages.


**PLEASE COMPLETE AND RETAIN FOR YOUR RECORDS**

Tray Easel TE- \_\_\_\_\_


Serial Number: \_\_\_\_\_

Purchase date: \_\_\_\_\_

# *TherAdapt*<sup>®</sup> Tray Easel (TE-300) Instructions

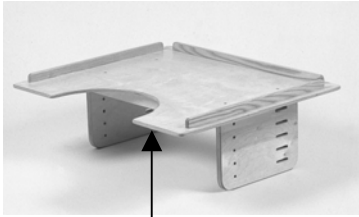

Front edge

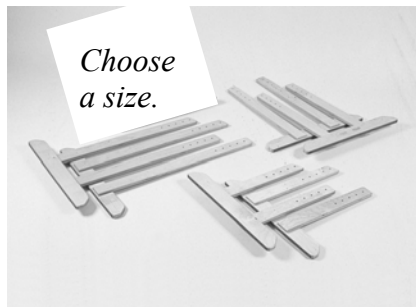

### Packing List:

- (1) Easel top component
- (2) Legs
- (4) 1 3/4" 1/4 -20 hex head bolts
- (4) Thru-knobs with 1/4 -20 threaded inserts
- (4) Nylon washers

### Attach the legs to the Easel top

The "angled" upright of the leg should be toward the front of the easel. Place a bolt through the desired hole/slot combinations to achieve the following height of the front edge to the floor. Place a nylon washer between the wood and knob. **Tighten all knobs securely before using.**



### Horizontal

	<u>LS-300L</u>	<u>LS-300M</u>	<u>LS-300T</u>
Top hole, Top slot	21"	28"	35"
2nd hole, 2nd slot	23 1/2"	30 1/2"	37 1/2"
Center hole, Center slot	26"	33"	40"
4th hole, 4th slot	28 1/2"	35 1/2"	42 1/2"
Bottom hole, Bottom slot	31"	38"	45"

### 10° Angle

Top hole, 2nd slot	20"	27"	34"
2nd hole, Center slot	22 1/2"	29 1/2"	36 1/2"
Center hole, 4th slot	25"	32"	39"
4th hole, Bottom slot	27 1/2"	34 1/2"	41 1/2"

### 25° Angle

Top hole, Center slot	19 1/2"	26 1/2"	33 1/2"
2nd hole, 4th slot	21 1/2"	28 1/2"	35 1/2"
Center hole, Bottom slot	23 1/2"	30 1/2"	37 1/2"

### 45° Angle

Top hole, 4th slot	18 1/2"	25 1/2"	32 1/2"
2nd hole, Bottom slot	20 1/2"	27 1/2"	34 1/2"



**FOR PROFESSIONAL ASSISTANCE CALL 1-800-261-4919**