



The therapeutic tricycle for children with disabilities



For questions or to place an order,
please contact:

AmTryke, LLC.

Phone: (888) - AMTRYKE ext. 14

Email: amtryke@ambucs.org

Visit our website: www.ambucs.org

Assembly Instructions

AmTryke Trainer
30-91-0339

The AmTryke Trainer is a revolutionary new and is designed to work specifically with the AmTryke models. Models that the trainer will work with are the AM-9S, AM-9XS, Snappy, AM-12, AM-16, Pro Series 1412, 1416, 1420, 2700 Series 20" and 24", 1424 and HP-1000.

Tools Needed
5mm Allen Wrench
10 mm Wrench

1) Remove bolts and screws from support riser.



2) Slide support riser over front (square tube) of roller system.




Revise Date: 3/3/2011


3) Insert bolts through support riser and thread nuts on back side and tighten.




4) The support riser can be adjusted for length of travel and height this varies based upon the model Amtryke.

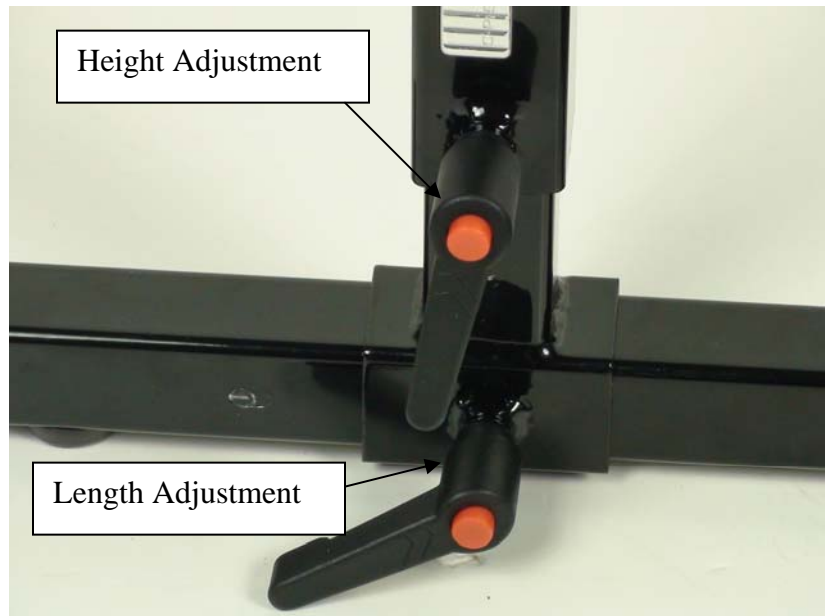

5) You will need to adjust the roller to fit the drive tire size. Loosen both knobs so that the size can be changed.

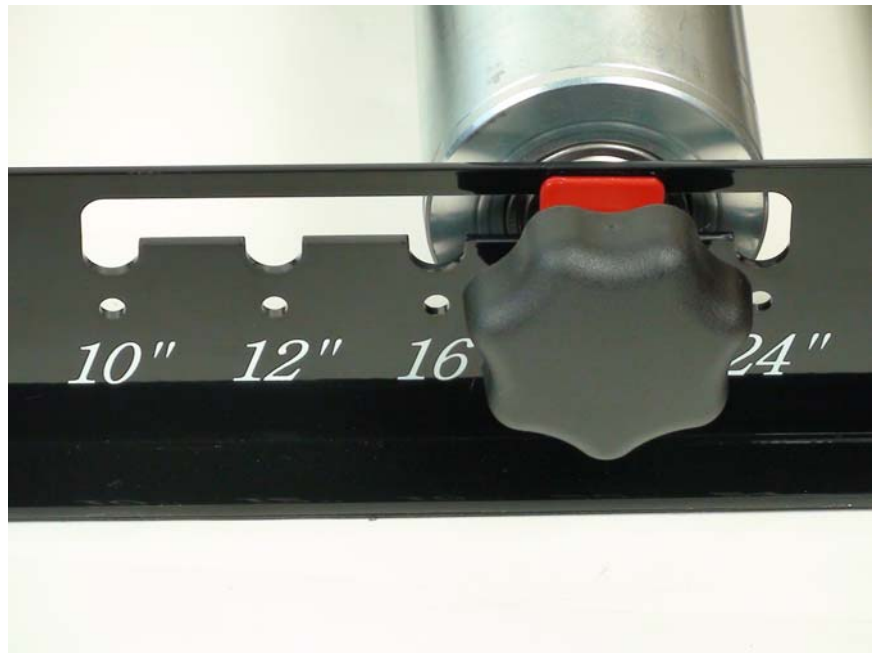

If you are unsure of your tire size please look on the sidewall of the tire as it will give a size there.

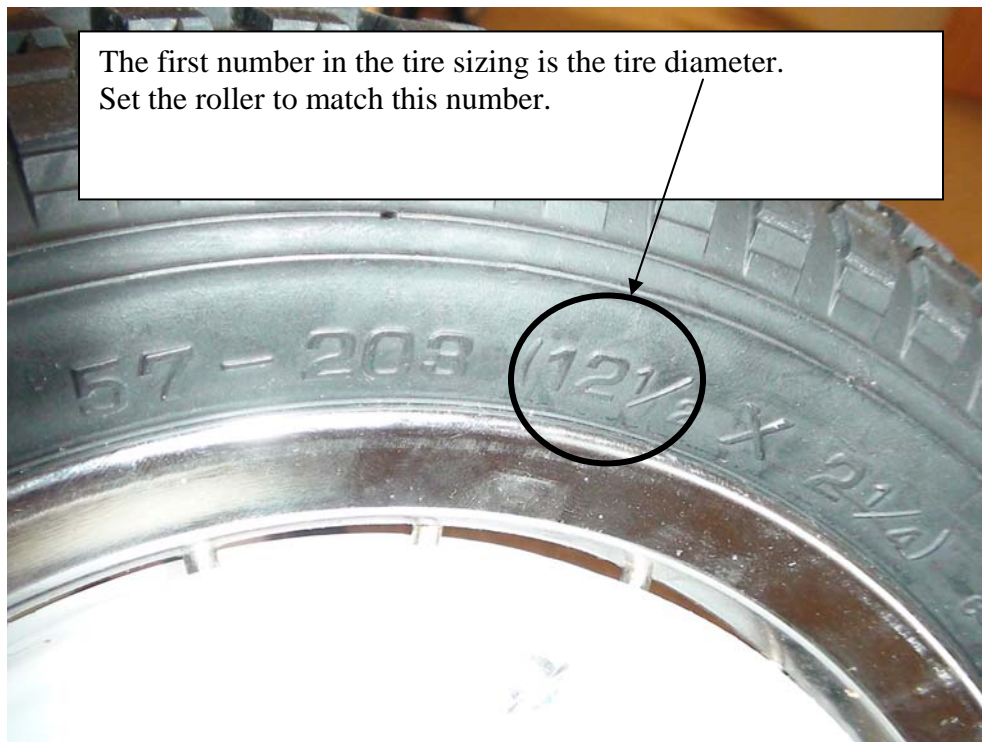

6) Included with the trainer are two Cyclops trainer blocks. The blocks are used under the non-drive wheels to support the tryke while using the trainer.



AM-12 on Trainer



AM-9 on Trainer

