

# Inclusion Chair

The **Inclusion Chair**, available in four sizes, serves as the ideal seating solution for mild to moderately involved users from preschool through college. This chair is an excellent choice when a standard classroom or desk chair does not provide sufficient support. The modular design of the chair allows it to be customized to the individual user without the look of a “therapeutic” chair.



The Inclusion Chair now features a height, depth and adjustable angle “drop” style seat made of durable, aesthetically appealing gray phenolic laminate. This feature allows the seat to back angle to be positioned between 75° and 95° for individualized therapeutic pelvic positioning and optimal spinal alignment. It can also be dropped down between the side frames to accept the extra thickness of a pressure relief cushion. A 2" wide, side release pelvic positioning belt is included to promote a stable base of support. The back and footplate, also made of phenolic, are predrilled and slotted for individual customizing options. The height and depth adjustable footplate is included for additional stabilization.

A rugged black 1" tubular steel frame, which allows for chair height and angle (10° tilt) adjustability now features adjustable height and angle glides at the bottom of the leg extension tubes for durability and the flexibility to adapt to uneven floors. Flip back armrests, now with comfortable padded arm pads, provide upper extremity support, assist with sit to/from stand, and allow the user to do a lateral transfer into and out of the chair if needed.



## FIT RIGHT GUIDE

See center insert page L

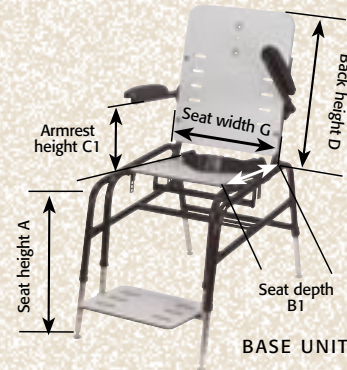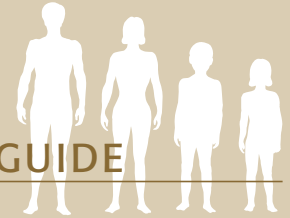

### Inclusion Chair

#### SMALL

##### IC-50 Inclusion Chair

Seat depth (B1):	10" - 14"
Seat width (G):	10" (5" or 8")*
Seat angle:	75° - 95°
Seat height to floor (A):	13" - 20"
Seat height to footplate (A):	10 1/2" - 17 1/2"
Back height (D):	16"
Back width (F):	12 3/4" (5" - 8")*
Armrest height (C1):	7" - 12 1/2"
Overall dimensions:	16 1/2"W x 31 1/2"L

##### IC-50AB Abduction Block

##### IC-50BK Butterfly Kit

##### IC-50FS Foot Sandals

Inside dimensions:	3"W x 5"L
Outside dimensions:	4 1/4"W x 5 1/2"L

##### IC-50HR Headrest

Back height (E):	22" - 27"
------------------	-----------

##### IC-50HS Heel Stop

##### IC-50PS Seat and Back Pad Set

##### IC-50PSW Seat (full width) and Back Pad Set

##### IC-50SB Support Blocks\* 5"H x 4"W (pair)

##### IC-50SK Support Kit

(includes FS, PS, 2 pair SB)

##### IC-50SW Head Support Wedges 6"H x 6"W (pair)

##### IC-50T Wood Tray

##### IC-50WK Locking Wheel Kit 4" H

#### MEDIUM

##### IC-150 Inclusion Chair

Seat depth (B1):	10" - 16"
Seat width (G):	13" (8" or 11")*
Seat angle:	75° - 95°
Seat height to floor (A):	13" - 20"
Seat height to footplate (A):	10 1/2" - 17 1/2"
Back height (D):	18"
Back width (F):	15 3/4" (8" - 11")*
Armrest height (C1):	7" - 12 1/2"
Overall dimensions:	19 1/2"W x 31 1/2"L

##### IC-150AB Abduction Block

##### IC-150BK Butterfly Kit

##### IC-150FS Foot Sandals

Inside dimensions:	3"W x 5"L
Outside dimensions:	4 1/4"W x 5 1/2"L

**MEDIUM (continued)****IC-150HR Headrest**

Back height (E): 24" - 29"

**IC-150HS Heel Stop****IC-150PS Seat and Back Pad Set****IC-150PSW Seat (full width) and Back Pad Set****IC-150SB Support Blocks\*** 5"H x 4"W (pair)**IC-150SK Support Kit**

(includes FS, PS, 2 pair SB)

**IC-150SW Head Support Wedges** 6"H x 6"W (pair)**IC-150T Wood Tray****IC-150WK Locking Wheel Kit** 4"H**LARGE****IC-250 Inclusion Chair**

Seat depth (B1): 10" - 18"

Seat width (G): 16" (11" or 14")\*

Seat angle: 75° - 95°

Seat height to floor (A): 16" - 23"

Seat height to footplate (A): 13 1/2" - 20 1/2"

Back height (D): 21"

Back width (F): 18 3/4" (11" - 14")\*

Armrest height (C1): 7" - 12 1/2"

Overall dimensions: 22 1/2"W x 31 1/2"L

**IC-250AB Abduction Block****IC-250BK Butterfly Kit****IC-250FS Foot Sandals**

Inside dimensions: 4"W x 8"L

Outside dimensions: 5 1/2"W x 8 1/2"L

**IC-250HR Headrest**

Back height (E): 27" - 32"

**IC-250HS Heel Stop****IC-250PS Seat and Back Pad Set****IC-250PSW Seat (full width) and Back Pad Set****IC-250SB Support Blocks\*** 5"H x 6"W (pair)**IC-250SK Support Kit**

(includes FS, PS, 2 pair SB)

**IC-250SW Head Support Wedges** 6"H x 6"W (pair)**IC-250T Wood Tray****IC-250WK Locking Wheel Kit** 4"H**X-LARGE****IC-350 Inclusion Chair**

Seat depth (B1): 10" - 18"

Seat width (G): 19" (14" or 17")\*

Seat angle: 75° - 95°

Seat height to floor (A): 16" - 23"

Seat height to footplate (A): 13 1/2" - 20 1/2"

Back height (D): 21"

Back width (F): 21 3/4" (14" - 17")\*

Armrest height (C1): 7" - 12 1/2"

Overall dimensions: 25 1/2"W x 31 1/2"L

**IC-350AB Abduction Block****IC-350BK Butterfly Kit****IC-350FS Foot Sandals**

Inside dimensions: 4"W x 8"L

Outside dimensions: 5 1/4"W x 8 1/2"L

**IC-350HR Headrest**

Back height (E): 27" - 32"

**IC-350HS Heel Stop****IC-350PS Seat and Back Pad Set****IC-350PSW Seat (full width) and Back Pad Set****IC-350SB Support Blocks\*** 5"H x 6"W (pair)**IC-350SK Support Kit**

(includes FS, PS, 2 pair SB)

**IC-350SW Head Support Wedges** 6"H x 6"W (pair)**IC-350T Wood Tray****IC-350WK Locking Wheel Kit** 4"H**\*Note:** Seat and Back width reduction is achieved by use of Support Blocks.

C.



D.



E.



F.



G.



H.



I.

**OPTIONS**

- A. A Heel Stop**, complete with dual ankle straps and curved side release buckles, is available to help keep the feet secure on the footplate. It is ideal for users who will be standing on the foot board and transferring into and out of the chair.
- B. A Wheel Kit**, complete with leg extensions and four casters, two of which have locking, swivel wheels, increases the portability of the chair from room to room, allowing the child to remain seated and well positioned for longer periods of time.
- C.** See pages 20-21 for these options shown: Abduction Block, Butterfly Kit, Foot Sandals, Headrest, Seat and Back Pad Set, Support Blocks (4), and Head Support Wedges.

**ACCESSORIES**

- D. A Comfortline Half Tray** (TH-100C) provides a comfortable padded upper extremity support surface that can be "flipped" up and out of the way or removed for transfers. For more information see page 67.
- E. A clear Viewline Half Tray** (TH-100V) allows the user full view of the lower body and surrounding area while still providing support for the upper extremity. For more information see page 67.
- F. A reversible Woodline Half Tray** (TH-100W) is an economical, easy to use upper extremity support surface that can be removed from the chair or simply flipped out of the way for transfers. For more information see page 67.
- G. An Enduroline Full Tray** features a melamine fiberboard tray with a moisture resistant top and a black PVC rim. For the IC-50 order TF-100E, for the IC-150 order the TF-100E for a 12" tray cutout or the TF-200E for a 15" tray cutout. For the IC-250 order the TF-300E, and for the IC-350 order the TF-400E. For additional information see page 67.
- H. The Viewline Full Tray** is a transparent polycarbonate tray that provides the user with a clear line of site to the lap and legs. It comes complete with a black PVC rim to keep items from rolling off the tray. For the IC-50 order TF-100V, for the IC-150 order the TF-100V for a 12" tray cutout or the TF-200V for a 15" tray cutout. For the IC-250 order the TF-300V, and for the IC-350 order the TF-400V. For additional information see page 67.
- I. A Wood Tray** is available with pencil rails and a cutout for comfort. It comes with standard "Z" shaped mounting hardware to allow the tray to slide on and off easily. For additional information see page 21.

# Options

Available for:  
**ODYSSEY CHAIRS**  
**TRANSITION CHAIRS**  
**INCLUSION CHAIRS**



**A. ABDUCTION BLOCK**

A padded, adjustable depth Abduction Block is available to provide medial thigh support for children who tend to cross their legs. (Odyssey Chairs, Transition Chairs, and Inclusion Chairs)

**B. ARMRESTS**

Adjustable height and angle Armrests are available to provide upper extremity support and weight bearing. When ordering a tray a pair of armrests must also be ordered to attach the tray. (Odyssey Chairs, and Transition Chairs) **Note:** Armrests come standard on the Inclusion Chairs.

**C. BUTTERFLY KIT**

A Butterfly Kit is available to provide anterior chest support. It attaches with straps around the back pad of the pad set and can be adjusted in any position along the back pad for a customized fit. **Note:** To be used with Pad Set or Pad Set – Wide.

**D. FOOT SANDALS**


Foot Sandals are available to secure the feet and can be placed in virtually any position needed on the footplate. They are made of molded plastic and come complete with two straps each with curved side release buckles for ease of use, comfort, and safety. (Odyssey Chairs, Transition Chairs, and Inclusion Chairs)

**E. HEADREST**

A 6" high adjustable Headrest is available for additional head and neck support. It attaches to the back of the chair and has 5" of vertical adjustability. The headrest comes complete with a posterior head pad and is slotted to accept a pair of head support wedges (see J). (Odyssey Chairs, Transition Chairs, and Inclusion Chairs)

**F. PAD SET**

The Pad Set consists of a seat and back pad made of washable vinyl. The seat pad is attached with Velcro®-like hook and loop while the back pad is secured with two bolts through the chair back. (Odyssey Chairs, Transition Chairs, and Inclusion Chairs)

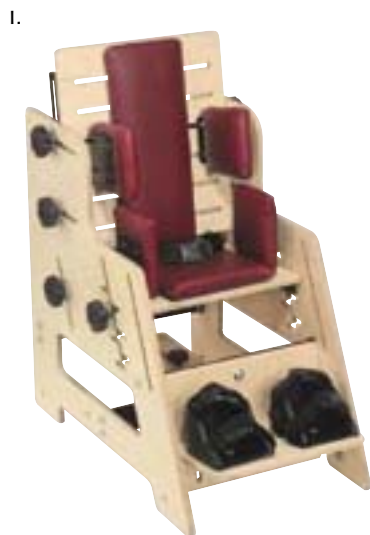

#### G. PAD SET-WIDE

The Pad Set-Wide features a full width seat pad and standard back pad made of washable vinyl. The seat pad is attached with Velcro®-like hook and loop while the back pad is secured with two bolts through the chair back. **Please note:** Support blocks cannot be used in the seat slots when the Pad Set – Wide is in place. Support block placement in the back slots is not affected. (Odyssey Chairs, Transition Chairs, and Inclusion Chairs)

#### H. SUPPORT BLOCKS

A pair of padded, vinyl covered Support Blocks are available to provide lateral pelvic, trunk, or thigh support, or to narrow the width of the chair. They attach to the seat or back of the chair with hardware provided and can be placed in any position necessary. Several Support Blocks may be used in customizing the fit of the chair for more involved users. (Odyssey Chairs, Transition Chairs, and Inclusion Chairs)

#### I. SUPPORT KIT

A Support Kit is available at a reduced cost. The kit includes a pair of Foot Sandals, a Seat and Back Pad Set, and two pairs of Support Blocks. (Odyssey Chairs, Transition Chairs, and Inclusion Chairs)

#### J. HEAD SUPPORT WEDGES

45° angle Head Support Wedges are available to provide lateral head control. They attach to the headrest to create a cradle and can be adjusted for a customized fit. The set comes complete with a pair of vinyl pads that attach to the wooden wedge shaped mounting brackets. (Odyssey Chairs, Transition Chairs, and Inclusion Chairs)

#### K. WOOD TRAY

A rugged Wood Tray is available for functional activities and provides additional upper extremity support and positioning. The spring loaded, quick release hardware on the Odyssey Chair and Transition Chair Trays allow for easy attachment to and removal from the armrests. **Please note:** Armrests must be ordered to attach the tray to these chairs. The Inclusion Chairs come with padded armrests that allow the Tray, with "Z" shaped mounting hardware, to slide on and off.

#### L. POLYCARBONATE TRAY

A clear Polycarbonate Tray is also available. The clear tray allows the user to see his or her lower extremities for greater body awareness and body scheme. The spring loaded, quick release hardware on the Odyssey Chair and Transition Chair Trays allows for easy attachment to and removal from the armrests. **Please note:** Armrests must be ordered to attach the tray to these chairs. **Also note:** The polycarbonate Odyssey Chair Tray is kidney shaped and cannot be used with other chairs.