
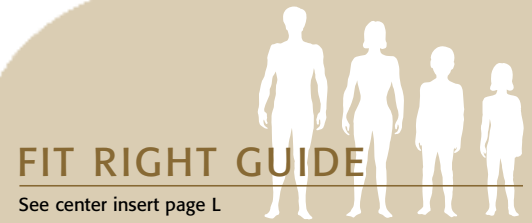
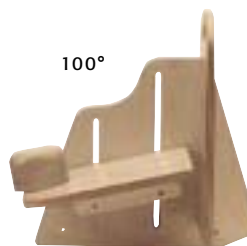
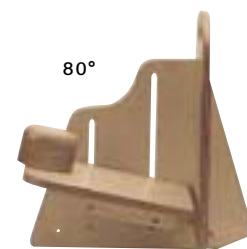


Adjustable Positioning Chair

The TherAdapt® Adjustable Positioning Chair offers three sizes of customized seating for the mild, moderate, or severely physically challenged user. Its wide range of accessories and back inserts allow individual customization to address differing levels of function. The chair is easily adjusted, allowing it to grow along with the child, and adapt as the child's specific needs change. The adjustability also allows the chair to be used by several different children in the classroom. It is a popular seating solution for activities such as circle-time, as well as during therapy sessions focused on feeding or table-top activities.


The seat is height and angle adjustable, and allows a seat to back angle range between 80° and 100°, to ensure proper pelvic positioning. The U-shaped side frames raise and lower the height of the chair, as well as angle the chair at 5°, 10°, or 20° tilt in space. The chair comes standard with a depth adjustable and removable abduction pommel and a pelvic positioning belt. The paired slots in the Adjustable Positioning Chair's back accept either the Corner Back Insert, Low Back Insert, Flat Back Insert, or Winged Back Insert. This customizing option offers head and/or trunk support for the more involved user.

BASE UNIT

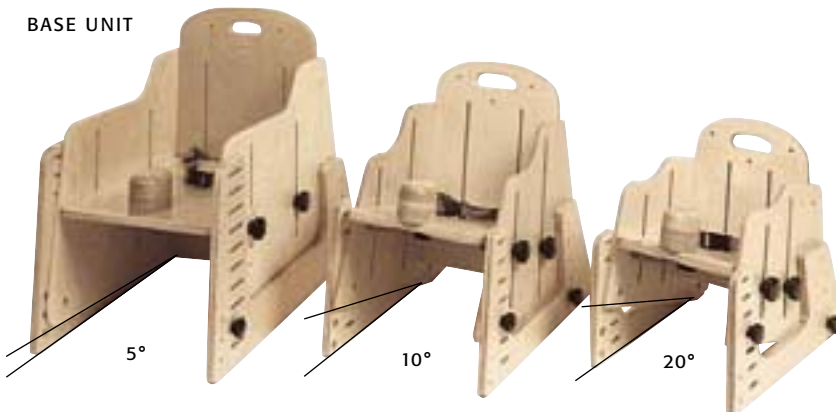

FIT RIGHT GUIDE

See center insert page L

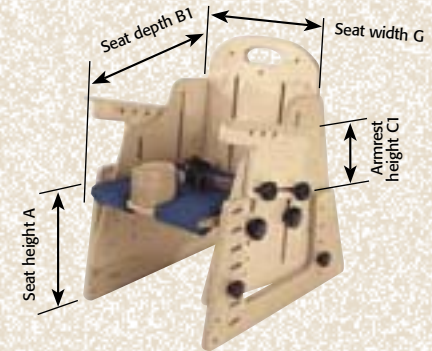

APC CHAIR WITH OPTIONAL ARMRESTS (OPTION A) AND PADDED SEAT PAD (OPTION C).

Adjustable Positioning Chair

EARLY INTERVENTION

APC-100 Adjustable Positioning Chair
 Seat depth (B1): 11 1/2" (6" - 11 1/2")*
 Seat width (G): 10" (6" - 10")*
 Seat height (A): 3" to 14"

APC-100AR Armrests (pair)
 Armrest height (C1): 4" - 11"

APC-100FP Footplate

APC-100P Padded Seat Pad

APC-100T Wood Tray**

Tray dimensions: 21 1/2" W x 20" D

Tray cutout (F): 11" W

APC-102AP 2" high Adduction Pads (pair)

APC-102BP 2" high Back Pad

APC-104BP 4" high Back Pad

APC-102SP 2" high Side Pads (pair)

APC-104SP 4" high Side Pads (pair)

PRESCHOOL/PRIMARY

APC-200 Adjustable Positioning Chair
 Seat depth (B1): 13 1/2" (8" - 13 1/2")*
 Seat width (G): 12" (8" - 12")*
 Seat height (A): 3" to 18"

APC-200AR Armrests (pair)

Armrest height (C1): 4 1/2" - 14"

APC-200FP Footplate

APC-200P Padded Seat Pad

APC-200T Wood Tray**

Tray dimensions: 24" W x 20" D

Tray cutout (F): 13" W

APC-202AP 2" high Adduction Pads (pair)

APC-202BP 2" high Back Pad

APC-204BP 4" high Back Pad

APC-202SP 2" high Side Pads (pair)

APC-204SP 4" high Side Pads (pair)

PRIMARY/INTERMEDIATE

APC-300 Adjustable Positioning Chair
 Seat depth (B1): 17 1/2" (12" - 17 1/2")*
 Seat width (G): 14" (8" - 14")*
 Seat height (A): 8" to 28"

APC-300AR Armrests (pair)

Armrest height (C1): 7" - 16"

APC-300FP Footplate

APC-300P Padded Seat Pad

APC-300T Wood Tray**

Tray dimensions: 26" W x 24" D

Tray cutout (F): 15" W

APC-303AP 3" high Adduction Pads (pair)

APC-303BP 3" high Back Pad

APC-305BP 5" high Back Pad

APC-303SP 3" high Side Pads (pair)

APC-305SP 5" high Side Pads (pair)

*Note: Seat depth and width reduction is achieved by use of an insert or back pads, and side or adduction pads.

**Note: Armrests must be ordered to attach the Tray to the chair.

OPTIONS

- A.** The **Armrests** are both height and angle adjustable for upper extremity support and positioning.
- B.** The **Footplate** is height, depth, and angle adjustable to provide a variety of positioning options for the feet. It comes with four straps that can be used as needed. The feet are secured with curved, side release buckles for ease in use.
- C.** The **Padded Seat Pad** is made of slip-resistant suede Naugahyde® and is attached with Velcro®-like hook and loop.
- D.** The paired **Adduction Pads** attach at the front slot on the side of the chair. They are used to control lower extremity positioning or to decrease the width of the chair when used in conjunction with the Corner Back Insert or a Winged Back Insert.
- E.** The paired **Side Pads** are available in two heights. They attach to the two slots on the side frame of the chair. The pads are used to reduce the width of the chair, provide lateral pelvic stability and femoral alignment. They can be used alone or in combination with the Low Back Insert or the Flat Back Insert, however, they cannot be used in conjunction with a Corner Back Insert or a Winged Back Insert.
- F.** The **Back Pads** are available in two different heights. They are used to reduce the seat depth of the chair. The pad is removable from the mounting support so that the length of the mounting bolts can be changed to any length required. It is recommended to use the smaller height pad at the sacrum for specific pelvic control, and the larger height pad up higher to create a new "back" to the chair. The Back Pads can also be used to "fill in" the space under Inserts (I through L) as they are raised to accommodate for growth.
- G.** A depth adjustable, quick release **Tray** is available to provide a large work surface and additional upper extremity support and positioning. (Armrests must be ordered to attach Tray to the chair.)

ACCESSORIES

- H. Mobile Base**
A **Mobile Base**, with locking caster wheels, is available to transport the Adjustable Positioning Chairs. The **APC-100**, with or without a Footplate, interfaces with **MB-A**. The **APC-200**, with or without a Footplate, interfaces with the **MB-B**. The **APC-300** with or without a Footplate, interfaces with the **MB-C**. Please note that the addition of a Mobile Base will raise the chair 3 1/2". For more information see page 93.
- I. Low Back Insert**
For information see page 12.
- J. Flat Back Insert**
For information see page 10.
- K. Corner Back Insert**
For information see page 12.
- L. Winged Back Insert**
For information see page 11.

