

TherAdapt[®] Products Inc.
(800) 261-4919 phone (866) 892-2478 fax

**Feeding Chair-
2-in-1 Booster Seat**

www.theradapt.com

Please read all of the following information before using this product.

TherAdapt[®] Products Inc. has designed and manufactured this product with the well being of the user and caregiver(s) in mind. It is our endeavor to provide products which are attractive, comfortable, useful, highly adjustable, long lasting and safe. As when using any product, precautions must be taken.



WARNING: This product must only be used under the recommendation of a physician and / or licensed therapist and with adult supervision. Never leave user unattended. It is your responsibility to properly use and maintain your Feeding Chair. Failure to follow USE and CARE instructions or ASSEMBLE the chair properly as outlined may result in bodily injury.

THE MANUFACTURER IS NOT RESPONSIBLE FOR INJURIES RESULTING FROM ANY CUSTOMIZATION OR MODIFICATION OF THE EQUIPMENT THAT WAS NOT COMPLETED BY THE MANUFACTURER.

USE: The Feeding Chair is designed for two purposes: as a freestanding floor chair or used secured atop a conventional stationary chair as a booster seat. **No other use is permitted**, including, but not limited to a seat used for transportation in an automobile, stroller, or any other mobility device.

CARE: Take care to inspect the ongoing condition of all parts of your Feeding Chair. Verify that all knobs, bolts, and washers are properly positioned and secure. Contact the manufacturer for replacement or repair of necessary items. All components may be cleaned with mild soap and water, or a gentle sanitizing spray. **Do not** use abrasive cleaners. Wipe components dry after cleaning. Always store indoors.

WARRANTY: The manufacturer provides a full one (1) year warranty, beginning the date shipped by the manufacturer. The warranty provides full replacement of all parts against breakage when used in compliance with the manufacturer's USE and CARE instructions. Use or modification beyond the normal capacity or range of adaptability designed into the products will void the warranty. The foregoing is TherAdapt's sole liability for consequential damages.

PLEASE COMPLETE AND RETAIN FOR YOUR RECORDS

Feeding Chair FC-200

Serial Number: _____

Purchase date: _____

TherAdapt[®] Feeding Chair (FC-200)- 2-in-1 Booster Seat Instructions

Packing List:

- (1) Chair base with a hinged rear leg with (1) 5/16-18 T-nut and (1) nylon washer
- (1) 1" wide pelvic strap with (1) side release buckle and (2) double slotted rings
- (2) 1" wide chair securement straps with (1) side release buckle and (1) double slotted ring each
- (1) Slotted wooden back with (2) black metal brackets and (6) 1/2" 10/32-24 Phillips head screws
- (1) Slotted wooden footrest with (2) 5/16- 18 T-nuts


- (4) 5/16-18 x 1" carriage bolts *
- (4) Large black knobs with threaded inserts *
- (2) Knobs with 5/16-18 x 1 1/2" stem bolts *
- (6) Nylon washers *

* Included in hardware bag

The 2-in-1 Booster Seat comes partially assembled. Once fully assembled it should be adjusted properly to the individual user to provide a customized fit.


1. Attach the back support.

Place the backrest between the sides of the chair with the brackets positioned on the backside of the support. Insert the (4) 5/16-18 x 1" carriage bolts through the brackets, then through the horizontal slots on the side of the chair. Place a nylon washer over each bolt and secure with a large knob with threaded insert. Tighten loosely until the chair is adjusted properly.

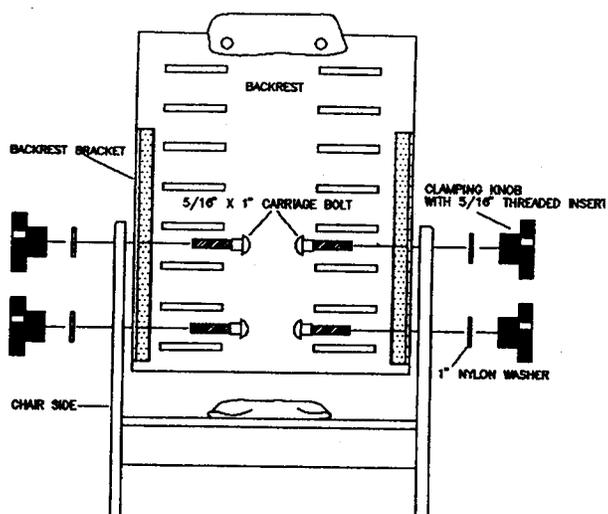

2. Attach the footrest.

Position the footrest with the support bar on the underside. Align the two holes in the footrest support bar with the two vertical slots in the calf rest. Insert the (2) knobs with stem bolts through the (2) nylon washers, through the vertical slots, into the holes in the support bar and tightened into the T-nuts. Tighten loosely until the chair is adjusted properly.

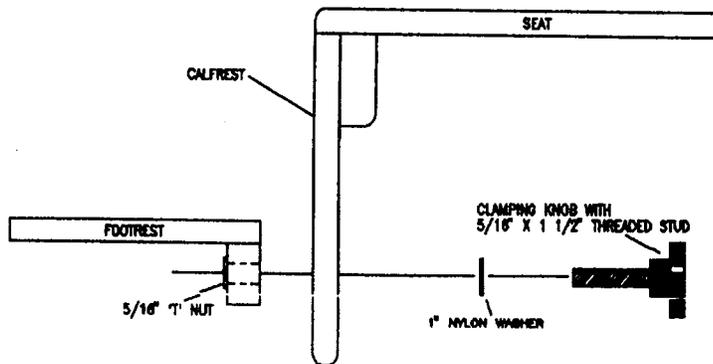


Once the chair is assembled it should fit the user properly.

1. Adjust the seat depth and back angle.

The seat should be 1 ½ “ – 2” behind the bend at the knee with the user sitting on the seat, bending forward to get the hips to the back of the chair. When used as a freestanding floor chair, take special care to **tighten the rear leg knob** (as shown).

- A. To adjust the seat depth, loosen the (4) large knobs that secure the back and move it forward or backward to achieve the desired depth. **Tighten the bottom two knobs securely.**
- B. To adjust the back angle, move the top of the seat back from vertical until the desired angle is achieved. **Tighten the top two knobs securely.**

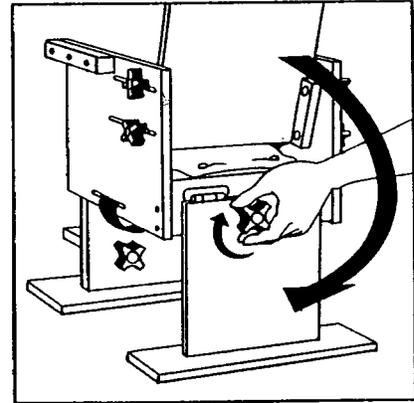


2. Adjust the pelvic positioning belt.

The 1” adjustable pelvic belt with side release buckle should be positioned in the notch created by the bend of the lap between the thigh and the pelvis, and tightened snugly. **Note:** The double slotted rings on each side of the pelvic strap have the loose ends of the strap threaded back through. This prevents the strap from slipping, makes the belt completely adjustable and allows the belt to snap closed at any location.

3. Adjust the footrest height.

Loosen the (2) knobs under the chair. Raise or lower the footrest so that the end of the thigh is resting securely on the front edge of the seat.

- 4. Secure the 2-in-1 Booster Seat** when used as a Booster Seat. When used as a booster seat, it is your responsibility to position the booster seat on a stationary chair that remains stable with the addition of the booster seat and child, and which affords no opportunity for slipping or tipping. To secure the booster seat, fasten the restraining strap around the rear legs of the conventional chair, below the seat. Wrap the restraining strap at least one full revolution around each rear leg and set the buckle between the rear legs. Pull the strap tight, and be sure there is sufficient tension on the restraining strap so as not to loosen during use.

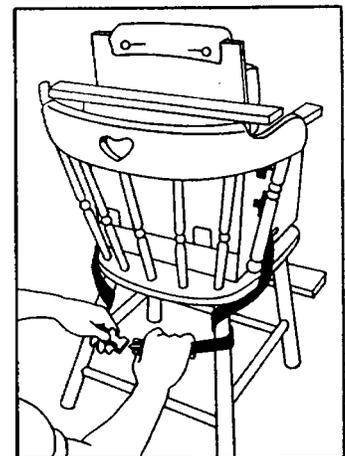

Attach chair Options purchased and follow instructions.

Note: Placement of options may affect previous adjustments. Re-assess steps 1 – 3 for a proper fit.

***TherAdapt*[®] Feeding Chair- 2-in-1 Booster Seat Options- Instructions**

1. Attach the Abduction Pommel. (FC-200AB)

Packing List:

(1) Padded pommel with metal “L” bracket

(2) Knobs with ¼ - 20 threaded bolts

Place the pommel on the padded seat pad with the metal bracket against the bottom side of the seat. Secure with (2) knobs with bolts into the (2) pre-installed ¼ - 20 threaded T-nuts on the seat.

2. Attach the Armrests (FC-200AR)

Packing List:

(2) Armrests

(4) 5/16 – 18 x 1 ¾” carriage bolts

(4) Nylon washers

(4) Thru-knobs with 5/16-18 threaded inserts

A. Insert (2) carriage bolts through the slots on side of the chair from the **inside**. Attach the armrest to the **outside** of the chair. Place a nylon washer over the bolt and secure with knobs. Repeat for other armrest.

B. Adjust to desired height and depth. **Tighten all knobs securely before using.**

3. Adjust the butterfly harness. (FC-200BK)

The butterfly comes attached to the chair with straps secured around the back pad. It should be adjusted such that the top of the strap is level with the shoulders. To do this: loosen (or remove) the back pad from the back of the chair by loosening (or removing) the (2) Phillips head screws on the backside of the chair. Slide the straps to the desired location on the pad and tighten (or replace) the Phillips head screws.

Note: The butterfly harness should be positioned on the chair so that the top of the butterfly straps is even with the shoulders and the buckle is below the collarbone. Adjust the length of the top straps to position the chest piece at the desired location on the chest (see below). Provide a “snug” fit by pulling on the lower set of straps only.

Care should be taken not to position the harness higher than the notch felt at the top of the sternum (breast bone) as strangulation could occur.

4. Attach the Foot Sandals. (FC-200FS)

Packing List:

(2) Foot sandals

(4) Foot straps with (1) side release buckle each


(4) ¼ -20 x 1” carriage bolts

(4) Thru-knobs with ¼ -20 threaded inserts

(4) Nylon washers

A. Attach the foot straps such that the longer strap is placed in the slots toward the heel of the foot sandal and the shorter strap is toward the toe of the foot sandal.

B. Attach the foot sandals to the footplate with (2) 1” carriage bolts and knobs with threaded inserts. Place a nylon washer between the footplate and the knob. **Tighten all knobs securely.**

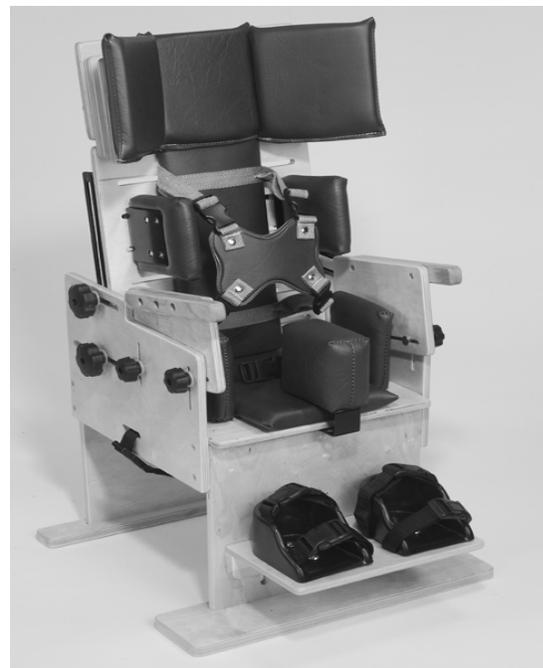

5. Attach the headrest. (FC-200HR)

Packing list:

- (1) "T" shaped headrest
- (1) Head pad with hook and loop fastener
- (2) Knobs with 5/16 - 18 threaded stem bolts

Secure the pad to the headrest. Remove the (1) self-adhering strip of loop fastener from the pad. Adhere the loop fastener to the top, front of the headrest. Position the pad on the headrest and secure. Using the (2) 5/16 stem knobs and (2) nylon washers (found in a baggie shrink wrapped to the headrest), pass them through the long slot in the headrest, then through the existing T-nuts in the back of the chair and secure.



6. Secure the padded seat. (FC-200PS or FC-200PSW)

Note: The padded back has already been attached. The padded seat pad attaches to the seat with hook and loop fastener. Remove the (1) self-adhering strip of loop fastener from the pad. Adhere the loop fastener to the top of the seat. Position the pad on the seat and secure the hook and loop fastener.

7. Attach the Support Blocks. (FC-200SB)

Packing List:

- (2) Support blocks with metal plates and 1/4-20 bolts
- (4) Nylon washers
- (4) Thru knobs with 1/4 - 20 inserts

Attach in desired set of paired slots on the back and / or seat. Place nylon washers on the bolts and secure with knobs. **Tighten securely before use.**



8. Attach the support wedges. (FC-200SW)

Packing list:

- (2) Wedge shaped blocks
- (2) Pads with hook and loop fastener
- (2) Knobs with 1/4 - 20 threaded bolts
- (2) Nylon washers


A. Secure the pads to the wedges. Remove the (1) self-adhering strip of loop fastener from each pad. Adhere the loop fastener to the front of the wedges and secure the hook and loop together.

B. Attach the wedges to the front of the headrest. Place the knobs with bolts through the desired slot on the backside of the headrest. Secure in the pre-installed T-nuts on the support wedge. Place a washer between the knob and the wood. **Tighten securely.**



9. Attach the tray. (FC-200T or FC-200TC)

Pull the quick release spring lock pins.
Slide the tray onto the armrests and release
the spring lock pins into one of the three
sets of holes on the armrest.

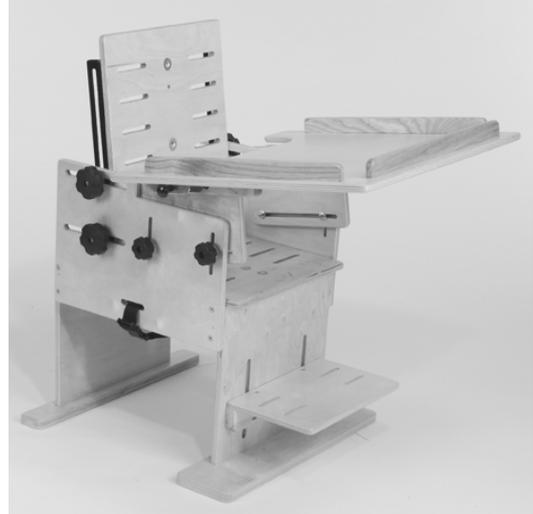

FOR PROFESSIONAL ASSISTANCE CALL 1-800-261-4919