

TherAdapt[®] Products Inc.
(800) 261-4919 phone (866) 892-2478 fax

**Adjustable Classroom
Chair**

www.theradapt.com

Please read all of the following information before using this product.

TherAdapt[®] Products Inc. has designed and manufactured this product with the well being of the user and caregiver(s) in mind. It is our endeavor to provide products which are attractive, comfortable, useful, highly adjustable, long lasting and safe. As when using any product, precautions must be taken.



WARNING: This product must only be used under the recommendation of a physician and / or licensed therapist and with adult supervision. Never leave user unattended. It is your responsibility to properly use and maintain your Adjustable Classroom Chair. Failure to follow USE and CARE instructions or ASSEMBLE the chair properly as outlined may result in bodily injury.

THE MANUFACTURER IS NOT RESPONSIBLE FOR INJURIES RESULTING FROM ANY CUSTOMIZATION OR MODIFICATION OF THE EQUIPMENT THAT WAS NOT COMPLETED BY THE MANUFACTURER.

USE: The Adjustable Classroom Chair is designed for a single purpose: as a freestanding chair. It is to be used on the floor only. **No other use is permitted**, including, but not limited to a seat used for transportation in an automobile, stroller, or any other mobility device. Extreme caution must be used when using the Adjustable Classroom Chair in conjunction with a TherAdapt[®] Mobile Base.

CARE: Take care to inspect the ongoing condition of all parts of your Adjustable Classroom Chair. Verify that all knobs, bolts, and washers are properly positioned and secure. Contact the manufacturer for replacement or repair of necessary items. All components may be cleaned with mild soap and water, or a gentle sanitizing spray. **Do not** use abrasive cleaners. Wipe wood components dry after cleaning. Excess amounts of liquid on the wood may damage the finish. Always store indoors.

WARRANTY: The manufacturer provides a full one (1) year warranty, beginning the date shipped by the manufacturer. The warranty provides full replacement of all parts against breakage when used in compliance with the manufacturer's USE and CARE instructions. Use or modification beyond the normal capacity or range of adaptability designed into the products will void the warranty. The foregoing is TherAdapt's sole liability for consequential damages.

PLEASE COMPLETE AND RETAIN FOR YOUR RECORDS

Adjustable Classroom Chair ACC-_____ **Serial Number:** _____

Purchase date: _____

***TherAdapt*[®] Adjustable Classroom Chair (ACC-100/200/300) Instructions**

Packing List:

- (1) Basic seat
- (1) 1" wide pelvic strap with (1) side release buckle and (2) double slotted rings
- (1) Knee block assembly with (2) vinyl kneepads
- (1) Small trapezoid end piece (front) and (1) large trapezoid end piece (rear)
- (2) 1 3/4" -20 carriage bolts
- (4) 1 3/4" -20 hex head bolts
- (6) Thru-knobs with 1/4"-20 inserts
- (6) Nylon washers

The basic Adjustable Classroom Chair comes partially assembled but should be adjusted properly to the individual user to provide a customized fit. Adjust the chair by following these steps:



- 1. Adjust the seat height and angle.** Remove the bolts and knobs from the front and back end pieces. **Note:** The bolts and knobs are put in **backwards** for shipping purposes. Select the height desired, and insert the bolts from the inside to the outside. Place nylon washers between the wood and the knobs. **Tighten the knobs securely.** **Note:** Using the same-paired holes on both the front and back supports creates a seat with a 10° forward slant. This is the recommended sitting angle for this chair, but other angles are possible by using other paired holes.



- 2. Adjust the pelvic positioning belt.** The 1" adjustable pelvic belt with side release buckle should be positioned in the notch created by the bend of the lap between the thigh and the pelvis, and tightened snugly. Note that the double slotted rings on each side of pelvic strap have the loose ends of the straps threaded back through. This prevents the strap from slipping, makes the belt completely adjustable and allows the belt to snap closed at any location.



3. Adjust the knee block depth.

The knee blocks are positioned by loosening the knobs under the front, top assembly and sliding them to the desired position. With the user's hips (buttocks) against the back block, the knees should be firmly cradled in the kneepads.

Note: The knobs attach underneath the seat to discourage tampering.



TherAdapt[®] Adjustable Classroom Chair Accessory Instructions



1. The Naugahyde[®] sacral pad slips over the sacral block and fastens with Velcro[®]-like hook and loop. Remove the self-adhering strip of loop fastener from the pad. Adhere the loop fastener to the bottom, rear of the sacral block. Slip the sacral pad over the sacral block and fasten the hook and loop.

2. The seat pad, constructed of suede Naugahyde[®] to stabilize the buttocks, also attaches with Velcro[®]-like hook and loop fastener. Remove the (3) self-adhering strips of loop fastener from the bottom of the pad. Adhere the loop fastener to the top of the seat. Position the seat pad on the seat and fasten the hook and loop fastener.

FOR PROFESSIONAL ASSISTANCE CALL 1-800-261-4919