

TherAdapt® Products Inc.

(800) 261-4919 phone (866) 892-2478 fax

www.theradapt.com

Adjustable Foot Stool

Please read all of the following information before using this product.

TherAdapt® Products Inc. has designed and manufactured this product with the well being of the user and caregiver(s) in mind. It is our endeavor to provide products which are attractive, comfortable, useful, highly adjustable, long lasting and safe. As when using any product, precautions must be taken.



WARNING: This product must only be used under the recommendation of a physician and / or licensed therapist and with adult supervision. Never leave user unattended. It is your responsibility to properly use and maintain your Adjustable Foot Stool. Failure to follow USE and CARE instructions or ASSEMBLE the Adjustable Foot Stool properly as outlined may result in bodily injury.

THE MANUFACTURER IS NOT RESPONSIBLE FOR INJURIES RESULTING FROM ANY CUSTOMIZATION OR MODIFICATION OF THE EQUIPMENT THAT WAS NOT COMPLETED BY THE MANUFACTURER.

USE: The Adjustable Foot Stool was designed for one purpose, as a foot support surface that may be used in conjunction with the TherAdapt® School Chairs or commercially available classroom, kitchen, or dining chairs. It is to be used on the floor only. **No other use is permitted.**

CARE: Take care to inspect the ongoing condition of all parts of your Adjustable Foot Stool. Contact the manufacturer for replacement or repair of necessary items. All components may be cleaned with mild soap and water, or a gentle sanitizing spray. **Do not** use abrasive cleaners. Wipe wood components dry after cleaning. Excess amounts of liquid on the wood may damage the finish. Always store indoors.

WARRANTY: The manufacturer provides a full one (1) year warranty, beginning the date shipped by the manufacturer. The warranty provides full replacement of all parts against breakage when used in compliance with the manufacturer's USE and CARE instructions. Use or modification beyond the normal capacity or range of adaptability designed into the products will void the warranty. The foregoing is TherAdapt's sole liability for consequential damages.

PLEASE COMPLETE AND RETAIN FOR YOUR RECORDS

Adjustable Foot Stool AFS-100 Serial Number: _____

Purchase date: _____

***TherAdapt* Adjustable Foot Stool (AFS-100) Instructions**

Packing List:

- (1) Adjustable Foot Stool with non-skid surface and metal bracket
- (4) nylon washers
- (4) knobs with a $\frac{1}{4}$ - 20 insert

No assembly required



For use with the **TherAdapt[®] School Chairs:**

1. Lengthen the School Chair legs to the required height for use at the table or desk.
2. Remove the knobs and washers from the Foot Stool and adjust top surface to 2", 4", or 6" from the floor, so that user's feet are flat on the non-skid surface.
3. Position chair legs in chair leg cutouts.

For use with a commercially available classroom, kitchen, or dining chair (with leg width of 11"-18") follow steps **2 and 3**.



FOR PROFESSIONAL ASSISTANCE CALL 1-800-261-4919