

TherAdapt[®] Products Inc.

(800) 261-4919 phone (866) 892-2478 fax

Adjustable Angle Easel

www.theradapt.com

Please read all of the following information before using this product.

TherAdapt[®] Products Inc. has designed and manufactured this product with the well being of the user and caregiver(s) in mind. It is our endeavor to provide products which are attractive, comfortable, useful, highly adjustable, long lasting and safe. As when using any product, precautions must be taken.



WARNING: This product must only be used under the recommendation of a physician and / or licensed therapist and with adult supervision. Never leave user unattended. It is your responsibility to properly use and maintain your Adjustable Angle Easel. Failure to follow USE and CARE instructions or ASSEMBLE the easel properly as outlined may result in bodily injury.

THE MANUFACTURER IS NOT RESPONSIBLE FOR INJURIES RESULTING FROM ANY CUSTOMIZATION OR MODIFICATION OF THE EQUIPMENT THAT WAS NOT COMPLETED BY THE MANUFACTURER.

USE: The Adjustable Angle Easel is designed for a single purpose: as a freestanding easel. It is to be used in conjunction with a stable support surface including, but not limited to a desk or table, with the bottom edge of the easel “hooked” over the support surface. **No other use is permitted.**

CARE: Take care to inspect the ongoing condition of all parts of your Adjustable Angle Easel. Verify that all knobs, bolts, and washers are properly positioned and secure. Contact the manufacturer for replacement or repair of necessary items. All components may be cleaned with mild soap and water, or a gentle sanitizing spray. **Do not** use abrasive cleaners. Wipe wood components dry after cleaning. Excess amounts of liquid on the wood may damage the finish. Always store indoors.

WARRANTY: The manufacturer provides a full one (1) year warranty, beginning the date shipped by the manufacturer. The warranty provides full replacement of all parts against breakage when used in compliance with the manufacturer’s USE and CARE instructions. Use or modification beyond the normal capacity or range of adaptability designed into the products will void the warranty. The foregoing is TherAdapt’s sole liability for consequential damages.

PLEASE COMPLETE AND RETAIN FOR YOUR RECORDS

Adjustable Angle Easel AAE-_____ **Serial Number:** _____

Purchase date: _____

***TherAdapt*[®] Adjustable Angle Easel (AAE-100) Instructions**

Packing List:

- (1) Easel top component with book ledge
- (2) Triangle side pieces with gray non-skid tape on bottom edge
- (4) 1 $\frac{3}{4}$ $\frac{1}{4}$ -20 hex head bolts
- (2) $\frac{1}{4}$ -20 locking nuts
- (2) Thru-knobs with $\frac{1}{4}$ -20 inserts
- (2) Nylon washers

This easel comes assembled but should be adjusted for individual use.



The front edge of this easel is designed to "hook" over the edge of the table or desk it is used on. The bottom edges of the easel have a non-skid surface for stability. To adjust the angle of the easel from 40° to 70° loosen the bolt/knob assembly on the sides of the easel and adjust to the desired angle.



Re-tighten knobs securely before using.

FOR PROFESSIONAL ASSISTANCE CALL 1-800-261-4919