

TherAdapt[®] Products Inc.
(800) 261-4919 phone (866) 892-2478 fax

Bolster Chair

www.theradapt.com

Please read all of the following information before using this product.

TherAdapt[®] Products Inc. has designed and manufactured this product with the well being of the user and caregiver(s) in mind. It is our endeavor to provide products which are attractive, comfortable, useful, highly adjustable, long lasting and safe. As when using any product, precautions must be taken.



WARNING: This product must only be used under the recommendation of a physician and / or licensed therapist and with adult supervision. Never leave user unattended. It is your responsibility to properly use and maintain your Bolster Chair. Failure to follow USE and CARE instructions or ASSEMBLE the chair properly as outlined may result in bodily injury.

THE MANUFACTURER IS NOT RESPONSIBLE FOR INJURIES RESULTING FROM ANY CUSTOMIZATION OR MODIFICATION OF THE EQUIPMENT THAT WAS NOT COMPLETED BY THE MANUFACTURER.

USE: The Bolster Chair is designed for a single purpose: as a freestanding chair. It is to be used on the floor only. **No other use is permitted**, including, but not limited to a seat used for transportation in an automobile, stroller, or any other mobility device. Extreme caution must be used when using the Bolster Chair in conjunction with a TherAdapt[®] Mobile Base.

CARE: Take care to inspect the ongoing condition of all parts of your Bolster Chair. Verify that all knobs, bolts, and washers are properly positioned and secure. Contact the manufacturer for replacement or repair of necessary items. All components may be cleaned with mild soap and water, or a gentle sanitizing spray. **Do not** use abrasive cleaners. Wipe wood components dry after cleaning. Excess amounts of liquid on the wood may damage the finish. Always store indoors.

WARRANTY: The manufacturer provides a full one (1) year warranty, beginning the date shipped by the manufacturer. The warranty provides full replacement of all parts against breakage when used in compliance with the manufacturer's USE and CARE instructions. Use or modification beyond the normal capacity or range of adaptability designed into the products will void the warranty. The foregoing is TherAdapt's sole liability for consequential damages.

PLEASE COMPLETE AND RETAIN FOR YOUR RECORDS

Bolster Chair BC- _____

Serial Number: _____

Purchase date: _____

TherAdapt[®] Bolster Chair (BC-100/200/300) Instructions

Packing List:

- (1) Basic chair with vinyl padded seat, base, and rectangular, slotted tray support
- (1) 1" wide pelvic strap with (1) side release buckle and (2) double slotted rings
- (1) Tray with (2) quick release spring loaded lock pins and (4) 1 1/2" 1/4 -20 carriage bolts
- (2) Foot straps of "loop" fastener
- (6) 1 3/4" 1/4 -20 hex head bolts
- (10) Thru-knobs with 1/4 -20 threaded inserts
- (10) Nylon washers

The basic Bolster Chair comes pre-assembled (except tray) but it should be adjusted properly to the individual user to provide a customized fit.



1. Adjust the seat height.

Remove the (4) knobs and bolts on the base. Adjust the seat to the appropriate height and re-install the bolts through the appropriate paired holes. Replace the knobs with a nylon washer between the knob and the wood.

Tighten securely.

Note: when using the optional footplate attach it to the base first, and then adjust the seat height for a proper fit.



- ### 2. Adjust the pelvic positioning belt.
- The 1" pelvic belt with side release buckle should be positioned in the notch created by the bend of the lap between the thigh and the pelvis, and tightened snugly. **Note:** The double slotted rings on each side of the pelvic strap have the loose ends threaded back through. This prevents the strap from slipping, makes the belt completely adjustable, and allows the belt to snap closed at any location.




3. Secure the feet / lower leg with straps

Place the loop strap in a figure "8" around the Lower leg and secure on the hook along the base of the chair.

4. Attach the tray and adjust.

- A. Adjust tray height by loosening the upper pair of knobs (A) and slide the inner, slotted, rectangular board into the appropriate niche. **Re-tighten knobs securely.**
- B. Place the front of the tray in the notch (B) on the rectangle tray support.

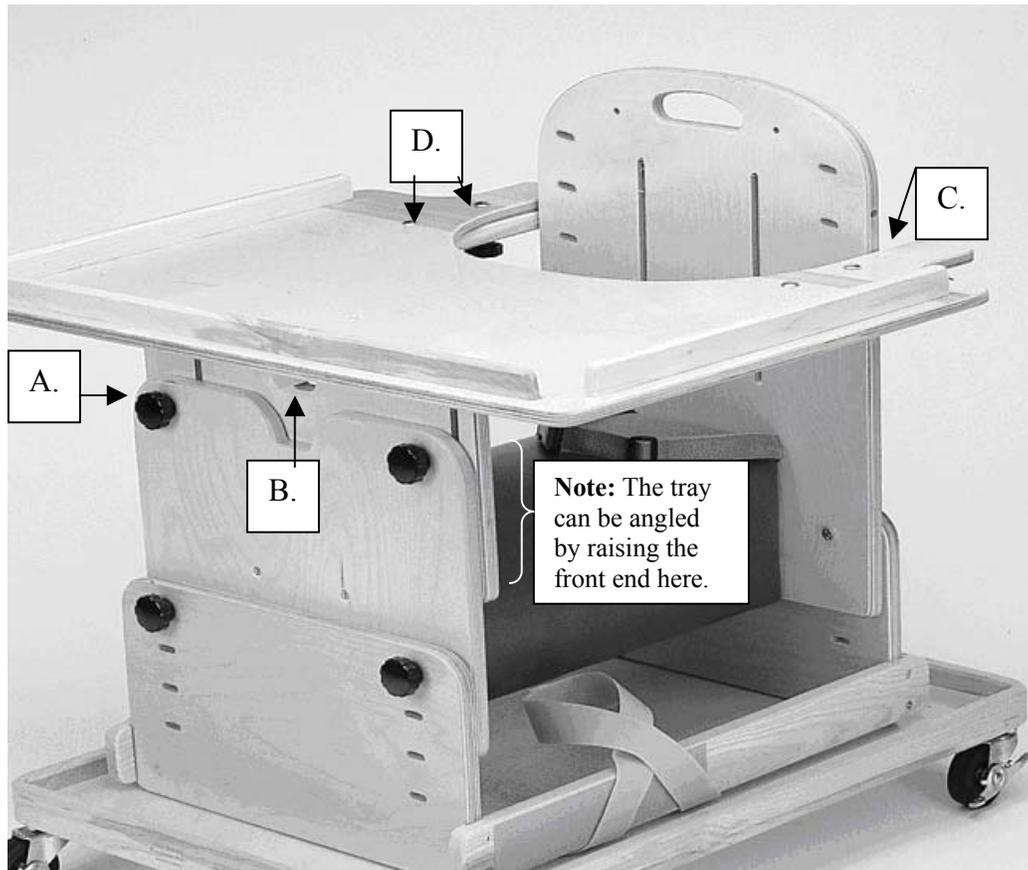

- C. Pull the quick release spring lock pins, line the tray up with the holes on the sides of the chair back (C), and release the pins. **Note:** The easiest way to attach or detach the tray is to stand behind the Bolster Chair.
- D. The depth of the tray is adjustable by loosening the knobs on the underside of the tray (D) and sliding the tray to the appropriate depth. **Re-tighten knobs securely before use.**

FOR PROFESSIONAL ASSISTANCE CALL 1-800-261-4919

***TherAdapt*[®] Bolster Chair Options- Instructions**

1. Attach the adjustable footplate. (BC-200FP, BC-300FP)

Packing List:

- (1) Footplate with (2) slotted metal brackets
- (2) Foot sandals with hook heel cups and loop cross straps
- (4) 1 3/4" 1/4 -20 hex head bolts
- (4) 1 1/4" 1/4 -20 carriage bolts
- (8) Thru-knobs with 1/4 -20 threaded inserts
- (8) Nylon washers

A. Attach each foot sandal to the footplate.


Place (2) hex bolts through the center slot on the sandal and attach through the slots on the footplate. Secure with knobs on the underside of the footplate. Place a nylon washer between the knob and the wood. **Tighten knobs securely before using.**

B. Attach footplate to the base of the chair.

Remove the loop fastener. Place (2) carriage bolts through the slotted metal bracket and through the holes on each side of the base. Secure with knobs. Place a nylon washer between the knob and hook fastener. **Tighten knobs securely before using.**



Note: When the Bolster Chair, with the footplate, is used on a TherAdapt[®] Mobile Base you must replace the (4) thru-knobs and (4) nylon washers with (4) metal washers and (4) 7/16" locking nuts which will be provided or available upon request.

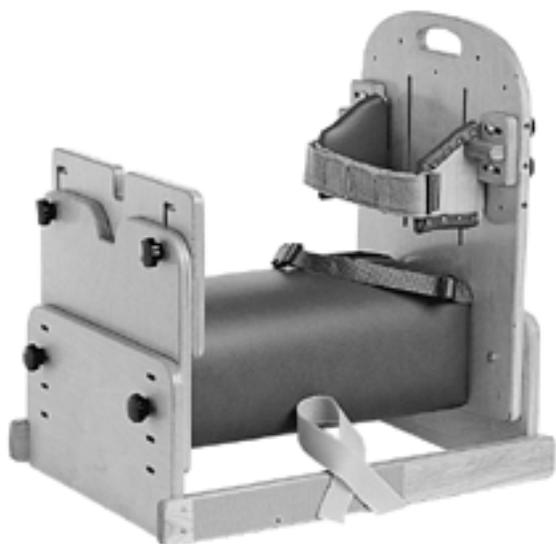

2. Attach the thoracic pads.

(BC-100TP, BC-200TP, BC-300TP)

Packing List:


- (2) Thoracic Pads with D-rings
- (1) Chest strap
- (4) Knobs with 1/4 -20 inserts
- (4) Nylon washers

Place the two hanger bolts on the rear of each pad support into the appropriate set of slots on the chair back. Before securing with knobs, place a nylon washer between the knob and the wood. Adjust the pads to the desired width and **tighten knobs securely.** Place the chest strap through the D-rings and secure hook and loop fastener.


***TherAdapt*[®] Bolster Chair Accessory Information**

Note: The back slots allow the *TherAdapt*[®] **Corner Back Insert**, **Low Back Insert**, **Flat Back Insert**, and **Winged Back Insert** to interface with the **Bolster Chair** providing head and/or trunk support as needed. The Bolster Chair Thoracic Pads cannot be used with an Insert.


Corner Back Insert

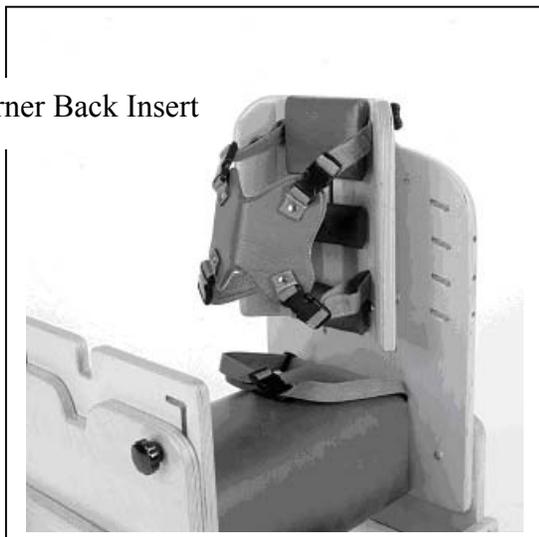


Low Back Insert

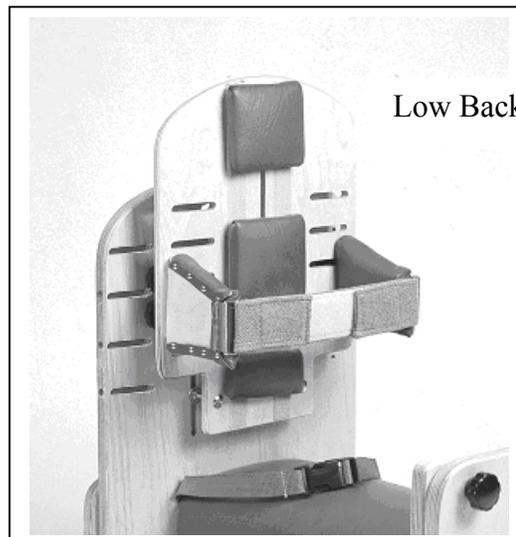

Winged Back Insert

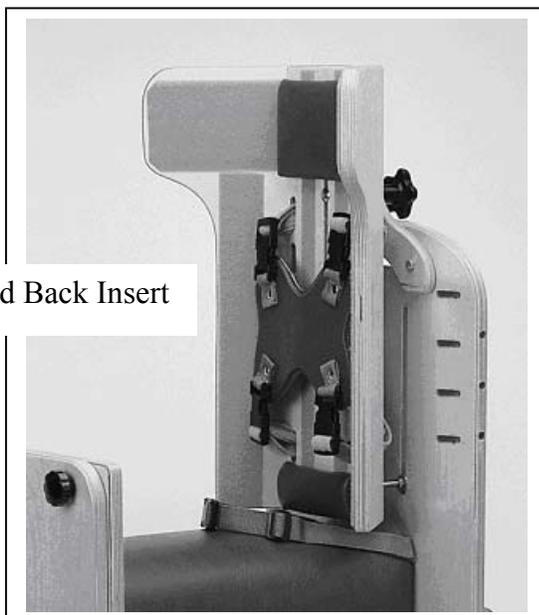

Flat Back Insert

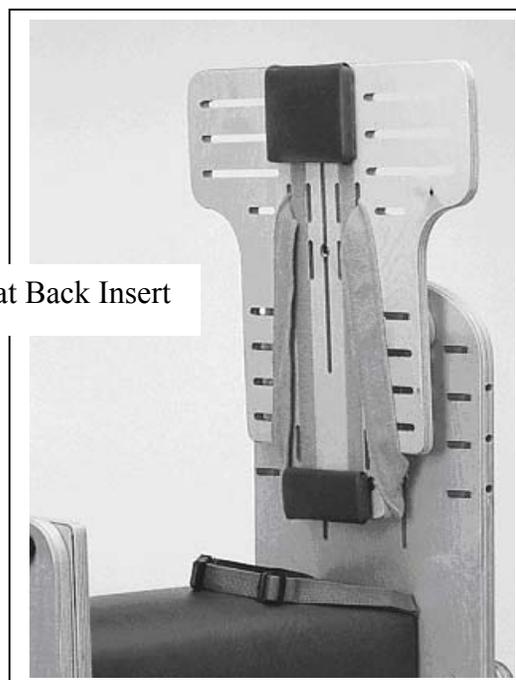

Please refer to the insert instructions provided when attaching an insert to the Bolster Chair.

FOR PROFESSIONAL ASSISTANCE CALL 1-800-261-4919

Use of the **Mobile Base** allows the chair to be moved safely about the home, clinic or classroom. **Caution** must be exercised when doing this and the casters should be locked immediately upon arrival at the new location.



For **BC-100** use **MB-A**
For **BC-200** use **MB-A**
For **BC-300** use **MB-B**

FOR PROFESSIONAL ASSISTANCE CALL 1-800-261-4919