

TherAdapt[®] Products Inc.

(800) 261-4919 phone (866) 892-2478 fax

www.theradapt.com

Extended Easel

Please read all of the following information before using this product.

TherAdapt[®] Products Inc. has designed and manufactured this product with the well being of the user and caregiver(s) in mind. It is our endeavor to provide products which are attractive, comfortable, useful, highly adjustable, long lasting and safe. As when using any product, precautions must be taken.



WARNING: This product must only be used under the recommendation of a physician and / or licensed therapist and with adult supervision. Never leave user unattended. It is your responsibility to properly use and maintain your Extended Easel. Failure to follow USE and CARE instructions or ASSEMBLE the easel properly as outlined may result in bodily injury.

THE MANUFACTURER IS NOT RESPONSIBLE FOR INJURIES RESULTING FROM ANY CUSTOMIZATION OR MODIFICATION OF THE EQUIPMENT THAT WAS NOT COMPLETED BY THE MANUFACTURER.

USE: The Extended Easel is designed for a single purpose: as a freestanding easel. It is to be used on the floor only. No other use is permitted.

CARE: Take care to inspect the ongoing condition of all parts of your Extended Easel. Verify that all knobs, bolts, and washers are properly positioned and secure. Contact the manufacturer for replacement or repair of necessary items. All components may be cleaned with mild soap and water, or a gentle sanitizing spray. Do not use abrasive cleaners. Wipe wood components dry after cleaning. Excess amounts of liquid on the wood may damage the finish. Always store indoors.

WARRANTY: The manufacturer provides a full one (1) year warranty, beginning the date shipped by the manufacturer. The warranty provides full replacement of all parts against breakage when used in compliance with the manufacturer's USE and CARE instructions. Use or modification beyond the normal capacity or range of adaptability designed into the products will void the warranty. The foregoing is TherAdapt's sole liability for consequential damages.

PLEASE COMPLETE AND RETAIN FOR YOUR RECORDS

Extended Easel EE- _____


Serial Number: _____

Purchase date: _____

TherAdapt[®] Extended Easel (EE-300) Instructions



Front edge

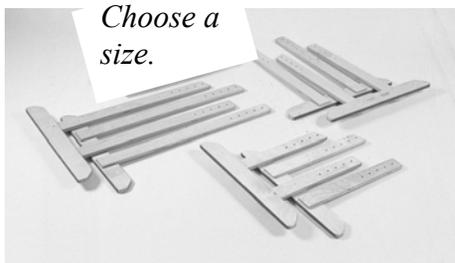

Packing List:

- (1) Easel top component
- (2) Legs
- (4) 1 3/4" 1/4 -20 hex head bolts
- (4) Thru-knobs with 1/4 -20 threaded inserts
- (4) Nylon washers

Attach the legs to the Easel top

The "angled" upright of the leg should be toward the front of the easel. Place a bolt through the desired hole/slot combinations to achieve the following height of the front edge to the floor. Place a nylon washer between the wood and knob. **Tighten all knobs securely before using.**



Note: The EE-300 has 4 holes and slots



	<u>LS-300L</u>	<u>LS-300M</u>	<u>LS-300T</u>
<u>Horizontal</u>			
Bottom hole, Top slot	25 1/2"	32 1/2"	39 1/2"
<u>10° Angle</u>			
3rd hole, Top slot	24 1/2"	31 1/2"	38 1/2"
Bottom hole, 2nd slot	27"	34"	41"
<u>25° Angle</u>			
Top hole, Top slot	18 1/4"	25 1/4"	32 1/4"
2nd hole, 2nd slot	20 3/4"	27 3/4"	32 1/4"
3rd hole, 3rd slot	23 1/4"	30 1/4"	37 1/4"
Bottom hole, Bottom slot	25 3/4"	32 3/4"	39 3/4"
<u>45° Angle</u>			
Top hole, 2nd slot	17 1/2"	24 1/2"	31 1/2"
2nd hole, 3rd slot	20"	27"	34"
3rd hole, Bottom slot	22 1/2"	29 1/2"	36 1/2"
<u>60° Angle</u>			
Top hole, 3rd slot	16 1/2"	23 1/2"	30 1/2"
2nd hole, Bottom slot	19"	26"	33"

FOR PROFESSIONAL ASSISTANCE CALL 1-800-261-4919