



## Assembly Instructions

### Foot Cups Installation Instructions

#### For use on AM-9S, AM-9XS, Snappy, AM-12 & AM-16

Part Number: 30-31-0149s Toddler, 30-31-0149m Medium

#### Contents of Foot Cups

Pair of Foot Cups

2 ea. 3" wide adhesive backed hook material

- 1) When installing on the AmTryke models listed above - remove the heel strap from the pedal plate.
- 2) Peel off the paper backing on the 3" wide adhesive backed hook material.
- 3) Apply the 3" wide adhesive hook material to the metal foot plate. Be careful not to cover the slots at the edges of the foot plate.
- 4) Re-install the heel strap through the bottom of the metal footplate.
- 5) Thread the heel straps and toe straps through the foot cups, push the foot cups securely against the hook material surface on the metal foot plate. Note; Line up heel plate with inner slot of medium size foot cup. Pull slack out of straps as you push firmly down on foot cup.

