

TherAdapt® Products Inc.

922 N. Washington Avenue, Ludington, MI 49431

Phone: 800-261-4919 • Fax: 866-892-2478

www.theradapt.com • customerservice@theradapt.com

School Chair Butterfly Kit Installation Instructions:

The straps to the butterfly kit for the school chair must be screwed into the frame of the chair back. The back does not need to be removed in order to do this. There is a $\frac{3}{4}$ inch piece of wood along the top and bottom of the chair back. The screws must be centered on this $\frac{3}{4}$ inch frame of the upholstered chair back. Each strap should be positioned within 1 to 1 $\frac{1}{2}$ inches of the side edge of the chair back.

Please see the pictures below for an example of strap placement from the back.





The positioning of the butterfly from the front:

