

TherAdapt[®] Products Inc.
(800) 261-4919 ph. (866)892-2478 fax

**Prone Stander
Tray Easel**

www.theradapt.com

Please read all of the following information before using this product.

TherAdapt[®] Products Inc. has designed and manufactured this product with the well being of the user and care giver(s) in mind. It is our endeavor to provide products which are attractive, comfortable, useful, highly adjustable, long lasting and safe. As when using any product, precautions must be taken.



WARNING: This product must only be used under the recommendation of a physician and / or licensed therapist and with adult supervision. Never leave user unattended. It is your responsibility to properly use and maintain your Prone Stander Tray Easel. Failure to follow **USE** and **CARE** instructions or **ASSEMBLE** the easel properly as outlined may result in bodily injury.

THE MANUFACTURER IS NOT RESPONSIBLE FOR INJURIES RESULTING FROM ANY CUSTOMIZATION OR MODIFICATION OF THE EQUIPMENT THAT WAS NOT COMPLETED BY THE MANUFACTURER.

USE: The Prone Stander Tray Easel is designed for a single purpose: as a freestanding easel. It is to be used on the floor only. **No other use is permitted.**

CARE: Take care to inspect the ongoing condition of all parts of your Prone Stander Tray Easel. Verify that all knobs, bolts, and washers are properly positioned and secure. Contact the manufacturer for replacement or repair of necessary items. All components may be cleaned with mild soap and water, or a gentle sanitizing spray. **Do not** use abrasive cleaners. Wipe wood components dry after cleaning. Excess amounts of liquid on the wood may damage the finish. Always store indoors.

WARRANTY: The manufacturer provides a full one (1) year warranty, beginning the date shipped by the manufacturer. The warranty provides full replacement of all parts against breakage when used in compliance with the manufacturer's **USE** and **CARE** instructions. Use or modification beyond the normal capacity or range of adaptability designed into the products will void the warranty. The foregoing is TherAdapt's sole liability for consequential damages.

PLEASE COMPLETE AND RETAIN FOR YOUR RECORDS

Prone Stander Tray Easel PS-_____ **-TE** **Serial Number:** _____

Purchase date: _____

***TherAdapt*® Prone Stander Tray Easel (PS-100TE/200TE/300TE) Instructions**

Packing List:

- (1)** Easel top component
- (2)** Legs
- (2)** 6” high rectangle shaped leg extensions (PS-100TE and PS-200TE only)
- (8)** 1 ¾” ¼ -20 hex head bolts
- (8)** Thru-knobs with ¼ -20 threaded inserts
- (8)** Nylon washers



Attach the legs to the Easel top

The “angled” upright of the leg should be toward the front of the easel. Place a bolt through the desired hole/slot combinations to achieve the following height of the front edge to the floor (see pages 2 & 3). Place a nylon washer between the wood and knob.

Tighten all knobs securely before using.



If additional tray height / angle is needed, attach the rectangle leg extension pieces to the tray sides, with the bottom holes below the bottom edge of the tray sides. Attach the legs to the 6” high extension pieces with the bolts and knobs provide.

FOR PROFESSIONAL ASSISTANCE CALL 1-800-261-4919

PS-100TE**LS-18****LS-18 + LS-6****Horizontal**

Top hole, Top slot	18"	24"
2nd hole, 2nd slot	19 1/2"	25 1/2"
Center hole, Center slot	21"	27"
4th hole, 4th slot	22 1/2"	28 1/2"
Bottom hole, Bottom slot	24"	30"

10° Angle

Top hole, 2nd slot	18"	24"
2nd hole, Center slot	19 1/2"	25 1/2"
Center hole, 4th slot	21"	27"
4th hole, Bottom slot	22 1/2"	28 1/2"

25° Angle

Top hole, Center slot	18"	24"
2nd hole, 4th slot	19 1/2"	25 1/2"
Center hole, Bottom slot	21"	27"

45° Angle

Top hole, 4th slot	18"	24"
2nd hole, Bottom slot	19 1/2"	25 1/2"

PS-200TE**LS-24****LS-24 + LS-6****Horizontal**

Top hole, Top slot	24"	30"
2nd hole, 2nd slot	25 1/2"	31 1/2"
Center hole, Center slot	27"	33"
4th hole, 4th slot	28 1/2"	34 1/2"
Bottom hole, Bottom slot	30"	36"

10° Angle

Top hole, 2nd slot	24"	30"
2nd hole, Center slot	25 1/2"	31 1/2"
Center hole, 4th slot	27"	33"
4th hole, Bottom slot	28 1/2"	34 1/2"

25° Angle

Top hole, Center slot	24"	30"
2nd hole, 4th slot	25 1/2"	31 1/2"
Center hole, Bottom slot	27"	33"

45° Angle

Top hole, 4th slot	24"	30"
2nd hole, Bottom slot	25 1/2"	31 1/2"

PS-300TE**LS-300T****Horizontal**

Top hole, Top slot	35"
2nd hole, 2nd slot	37 1/2"
Center hole, Center slot	40"
4th hole, 4th slot	42 1/2"
Bottom hole, Bottom slot	45"

10° Angle

Top hole, 2nd slot	34"
2nd hole, Center slot	36 1/2"
Center hole, 4th slot	39"
4th hole, Bottom slot	41 1/2"

25° Angle

Top hole, Center slot	33 1/2"
2nd hole, 4th slot	35 1/2"
Center hole, Bottom slot	37 1/2"

45° Angle

Top hole, 4th slot	32 1/2"
2nd hole, Bottom slot	34 1/2"

FOR PROFESSIONAL ASSISTANCE CALL 1-800-261-4919