

# TFH WHEELCHAIR PLATFORM SWING Wheelchair Restraint Instructions

2017

## Attaching Unwin Wheelchair Restraints to TFH Wheelchair Platform Swing

Follow these instructions for most wheelchairs. Read through these instructions completely before installing.

1. Position wheelchair on TFH Wheelchair Platform Swing facing front centrally over the rails and apply brakes. Front wheels should be touching front bumper
2. To attach the front straps, press on the ribbed part of the ATF and push firmly towards the wheelchair until the yellow clip drops and locks into the rail. Install these as close to the front of the Platform Swing as possible. Install the second clip. The position of this must be in line with the first.

**IMPORTANT: Check the plungers are fully engaged on both sides.**

3. Wrap the front straps around the wheelchair frame and through the back wheels (Fig 1) and connect the tongue & buckle.

**FIGURE 1**



# TFH WHEELCHAIR PLATFORM SWING Wheelchair Restraint Instructions

2017

4. Attach the rear straps similar to the front except the **yellow plungers must face the rear**. (Fig 2)

FIGURE 2



5. Release the wheelchair brake.
6. Wrap the rear straps in an X-pattern to the frame of the wheelchair and connect the tongue & buckle. (Fig 3)

FIGURE 3



REAR VIEW

FRONT VIEW

7. Pull the webbing through the over-center buckle until tight and close the buckle completely.
8. Resecure the Velcro.

## **IMPORTANT:**

1. **THE WHEELCHAIR SHOULD BE ATTACHED TIGHTLY TO THE WHEELCHAIR PLATFORM SWING. THERE SHOULD BE NO MOVEMENT OF THE WHEELCHAIR ON THE PLATFORM.**
2. **STRAP AND ATTACHMENT CONFIGURATIONS WILL VARY BY WHEELCHAIR. PLEASE CALL US WITH QUESTIONS.**